


Rehak Impulse™

Promoting
Independent
Lifestyle

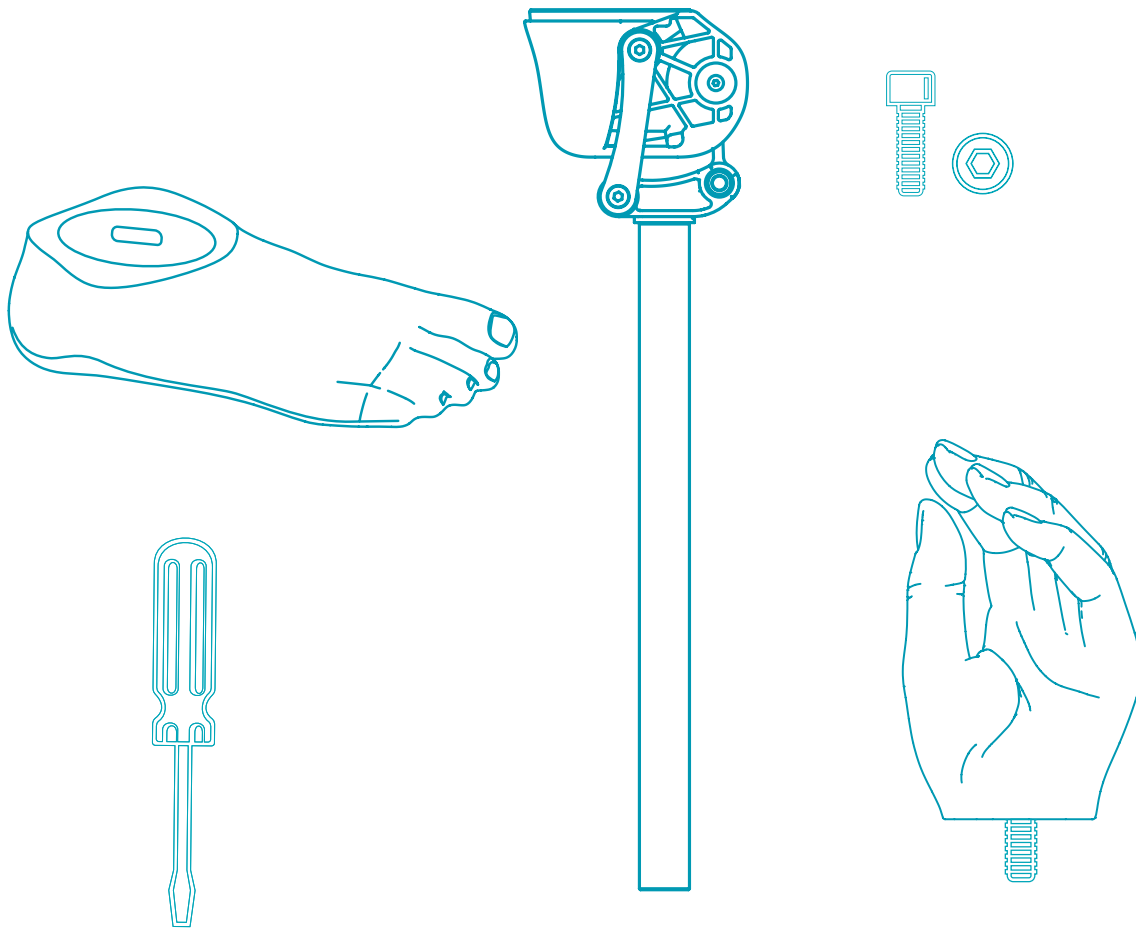

Catalogue

Volume II • Issue I



Erbil, ICRC Physical Rehabilitation Centre. A man who lost his legs when he drove his car over a landmine in Mosul in 2014 is being assisted by a physiotherapist during the rehabilitation process with this prosthesis at the PRC.
Copyright: RC Finland
Photographer: MANSIKKAMÄKI, Saara



ICRC

ICRC

THE INTERNATIONAL COMMITTEE OF THE RED CROSS

QUALITY REHABILITATION ACTIVITIES AND ASSISTIVE TECHNOLOGY FOR PERSONS WITH PHYSICAL DISABILITIES

The ICRC Physical Rehabilitation Programme (PRP) has been working for 40 years in countries affected by armed conflicts and other situations of violence to provide access to assistive products and rehabilitation services for people with physical disabilities. To meet these needs, the humanitarian institution has made rehabilitation devices more readily available by developing and propagating low-cost, high quality components for prosthetics and orthotics. This technology has become a standard in low- and middle-income countries and is being used by many institutions and organizations. As a result, service users have eased access to affordable and quality assistive devices.

“ propagating low-cost, high quality components for prosthetics and orthotics”

Over the past decades, the ICRC has invested a significant budget in R&D and industrial tools to have these products manufactured and commercialised by a swiss industrial partner. Following the successful research and development of the new dynamic prosthetic foot made in collaboration with the EPFL, the ICRC decided to launch a new working model in col-

laboration with the swiss-based Fondation Alfaset, located in La Chaux-de-Fonds (Neuchâtel).

From this new working model was born Rehab'Impulse, which aims to offer and develop high-quality components at accessible prices for people with disabilities in all regions of the world, enabling them to develop their full potential in the long term.



FONDATION ALFASET



“made by people with disabilities for people with disabilities”

OUR MISSION

We are a nonprofit organisation, offering socio-professional support by providing workplaces across various fields as well as adapted housing to people with disabilities. Our partners are the state actors in the field of

social welfare and our industrial and private customers. We are driven to support our beneficiaries in their social inclusion and professional integration taking into account their individual challenges.

INDUSTRIAL SERVICES

Our businesses offer a wide range of services, ranging from occupational work without performance requirements to industrial production, requiring advanced capabilities and respect for quality requirements and time commitments. In our workshops, people with disabilities find a workplace adapted to their individual abilities.

HELP AND SUPPORT

We offer employees with disabilities an infrastructure adapted to their needs and to their degree of autonomy. Our company currently offers adapted workplaces to almost 500 people on four production sites.

INTEGRATE AND VALUE

We tailor jobs and services to the individual needs and to the residual working capacity of people with disabilities, while ensuring quality services to our customers. We enable our employees to improve their moral, material and social situation and thus promote their integration into society and into the primary labour market.

COACH AND ACCOMPANY

For employees in need of support in their everyday activities, our multidisciplinary teams offer socio-educational support, personalized assistance and individual follow-up in their life project. To respect their different degrees of autonomy, we provide different types of housing to our beneficiaries, such as assisted housing with full board, adapted apartments or independent apartments with limited follow-up.

QUALITY

As your industrial partner and trusted supplier, we guarantee best in class quality and reliability in all our services.

Fondation Alfaset is certified ISO 9001:2015





Rumbek, Physical Rehabilitation Centre, Makur Madof
Diet, 46 years old, is both a service user and a staff
member. His left leg was amputated after being injured by
gunshot in 1998.
Copyright: ICRC
Photographer: MORTVEDT, Mari Aftret

REHAB'IMPULSE

PROMOTING INDEPENDENT LIFESTYLE

Rehab'Impulse is a Swiss brand, owned by the ICRC and launched in close collaboration with Fondation Alfaset, a non-profit organization, providing evolutionary, appropriate and high-quality components for mobility-enabling assistive technology, leaving no one behind.

**“more than 25
years in the field”**

The components of Swiss quality have been recognised by international standards and successfully used by persons with disabilities all over the world for over 25 years.

The products are provided at competitive prices, meeting the needs of both service users and providers in low- and middle-income countries, enabling the first step towards social inclusion.

OUR MISSION

We provide evolutionary, appropriate and high-quality components for mobility-enabling assistive technology, leaving no one behind.

OUR VISION

Our products contribute to persons with physical disabilities reaching their full potential in society.




GLOBAL PROVIDER OF ORTHOPAEDIC SOLUTIONS :

Rehab'Impulse does not only provide a product, it is mainly a global service for ortho-prosthetists. The range of services we offer is not exhaustive:

- Prostheses
- Materials
- Tools
- Machines
- Production solution consulting
- Logistics management

**“a solution for
your client and
your workshop”**



-  Fondation Alfaset - La Chaux-de-Fonds Switzerland
-  Rehab'Impulse direct shipments
-  ICRC supported structures distributing Rehab'Impulse products


51 countries served since 2020



> 40'000
prosthetic feet distributed in 2022

> 30'000
alignment kits distributed in 2022

> 42'000
users fitted with a Rehab'Impulse
device in 2022

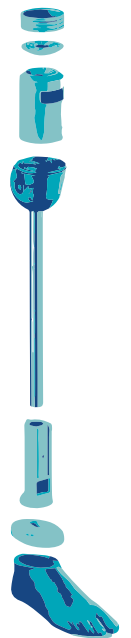

Products

FEET	12
TRANSTIBIAL ALIGNMENT	16
TRANSFEMORAL ALIGNMENT	22
KNEE JOINTS	26
UPPER LIMBS	34
COMPONENTS	38
RAW MATERIALS	42
HAND TOOLS	44
POWER TOOLS & EQUIPMENT	60
HARDWARE & CONSUMABLES	65
SPARE PARTS & ACCESSORIES	94
MOBILITY AIDS	104
PHYSICAL REHABILITATION AIDS	105
GLOSSARY	117
CONTACT	119

FEET

Foot SACH 2.0, Adult

FUNCTIONS AND BENEFITS

The Rehab'Impulse Prosthetic Foot SACH 2.0 (Solid-Ankle Cushion-Heel) is made of soft thermoplastic polyurethane moulded over a POM core. It is designed to provide the user with a cushioned heel and a flexible keel that offers a smooth rolling action during stance phase. The cosmetic toe allows wearing of sandals.

INTENDED USE

This product is intended for prosthetic fitting on adults with activity levels K1 (household ambulator) and K2 (limited community ambulator).

COMPATIBILITY

The Rehab'Impulse Foot SACH 2.0 is compatible with standard modular technology and with the Rehab'Impulse alignment system:

- 74-00137 Transtibial Alignment Kit, Adult
- 74-00134 Transfemoral Alignment Kit, Adult
- 74-00229 Transtibial Alignment Kit, Adult, Long Stump
- 74-00227 Transfemoral Alignment Kit, Child, Connector for Adult Foot

INCLUDED IN DELIVERY

- 1x Foot SACH 2.0, Adult
- 1x Socket head cap screw, stainless steel, M10x80mm
- 1x Ribbed lock washer M10



Terra



Olive

Article	Size	Heel height	Side	Colour	User weight
74-00139	22	10 mm	LEFT	OLIVE	≤ 80 kg (P4)
74-00141	22	10 mm	RIGHT	OLIVE	≤ 80 kg (P4)
74-00147	23	10 mm	LEFT	OLIVE	≤ 80 kg (P4)
74-00149	23	10 mm	RIGHT	OLIVE	≤ 80 kg (P4)
74-00087	24	10 mm	LEFT	OLIVE	≤ 80 kg (P4)
74-00089	24	10 mm	RIGHT	OLIVE	≤ 80 kg (P4)
74-00095	25	10 mm	LEFT	OLIVE	≤ 80 kg (P4)
74-00097	25	10 mm	RIGHT	OLIVE	≤ 80 kg (P4)
74-00103	26	10 mm	LEFT	OLIVE	≤ 80 kg (P4)
74-00106	26	10 mm	RIGHT	OLIVE	≤ 80 kg (P4)
74-00112	27	10 mm	LEFT	OLIVE	≤ 100 kg (P5)
74-00114	27	10 mm	RIGHT	OLIVE	≤ 100 kg (P5)
74-00120	28	10 mm	LEFT	OLIVE	≤ 100 kg (P5)
74-00122	28	10 mm	RIGHT	OLIVE	≤ 100 kg (P5)

FEET

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Terra



Olive

Article	Size	Heel height	Side	Colour	User weight
74-00143	22	10 mm	LEFT	TERRA	≤ 80 kg (P4)
74-00145	22	10 mm	RIGHT	TERRA	≤ 80 kg (P4)
74-00151	23	10 mm	LEFT	TERRA	≤ 80 kg (P4)
74-00153	23	10 mm	RIGHT	TERRA	≤ 80 kg (P4)
74-00091	24	10 mm	LEFT	TERRA	≤ 80 kg (P4)
74-00093	24	10 mm	RIGHT	TERRA	≤ 80 kg (P4)
74-00099	25	10 mm	LEFT	TERRA	≤ 80 kg (P4)
74-00101	25	10 mm	RIGHT	TERRA	≤ 80 kg (P4)
74-00108	26	10 mm	LEFT	TERRA	≤ 80 kg (P4)
74-00110	26	10 mm	RIGHT	TERRA	≤ 80 kg (P4)
74-00116	27	10 mm	LEFT	TERRA	≤ 100 kg (P5)
74-00118	27	10 mm	RIGHT	TERRA	≤ 100 kg (P5)
74-00124	28	10 mm	LEFT	TERRA	≤ 100 kg (P5)
74-00126	28	10 mm	RIGHT	TERRA	≤ 100 kg (P5)

FEET

Foot SACH 2.0, Child

FUNCTIONS AND BENEFITS

The Rehab'Impulse Prosthetic Foot SACH 2.0 (Solid-Ankle Cushion-Heel) is made of soft thermoplastic polyurethane moulded over a POM core. It is designed to provide the user with a cushioned heel and a flexible keel that offers a smooth rolling action during stance phase. The cosmetic toe allows wearing of sandals and gives the foot a natural appearance.

INTENDED USE

This product is intended for prosthetic fitting on children with activity levels K1 (household ambulator) and K2 (limited community ambulator).

COMPATIBILITY

The Rehab'Impulse Foot SACH 2.0 is compatible with standard modular technology and with the Rehab'Impulse alignment system.

- 74-00221 Transtibial Alignment Kit, Child
- 74-00223 Transfemoral Alignment Kit, Child

INCLUDED DELIVERY

- 1x Foot SACH 2.0, Child
- 1x Socket head cap screw, stainless steel, M8x80mm
- 1x Ribbed lock washer M8



Terra



Olive

Article	Size	Heel height	Side	Colour	User weight
74-00157	14	5 mm	LEFT	OLIVE	≤ 45 kg (P2)
74-00159	14	5 mm	RIGHT	OLIVE	≤ 45 kg (P2)
74-00165	15	5 mm	LEFT	OLIVE	≤ 45 kg (P2)
74-00167	15	5 mm	RIGHT	OLIVE	≤ 45 kg (P2)
74-00173	16	10 mm	LEFT	OLIVE	≤ 45 kg (P2)
74-00175	16	10 mm	RIGHT	OLIVE	≤ 45 kg (P2)
74-00181	17	10 mm	LEFT	OLIVE	≤ 45 kg (P2)
74-00183	17	10 mm	RIGHT	OLIVE	≤ 45 kg (P2)
74-00189	18	10 mm	LEFT	OLIVE	≤ 60 kg (P3)
74-00191	18	10 mm	RIGHT	OLIVE	≤ 60 kg (P3)
74-00197	19	10 mm	LEFT	OLIVE	≤ 60 kg (P3)
74-00199	19	10 mm	RIGHT	OLIVE	≤ 60 kg (P3)
74-00205	20	10 mm	LEFT	OLIVE	≤ 60 kg (P3)
74-00207	20	10 mm	RIGHT	OLIVE	≤ 60 kg (P3)
74-00213	21	10 mm	LEFT	OLIVE	≤ 60 kg (P3)
74-00215	21	10 mm	RIGHT	TERRA	≤ 60 kg (P3)

FEET

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Terra



Olive

Article	Size	Heel height	Side	Colour	User weight
74-00161	14	5 mm	LEFT	TERRA	≤ 45 kg (P2)
74-00163	14	5 mm	RIGHT	TERRA	≤ 45 kg (P2)
74-00169	15	5 mm	LEFT	TERRA	≤ 45 kg (P2)
74-00171	15	5 mm	RIGHT	TERRA	≤ 45 kg (P2)
74-00177	16	10 mm	LEFT	TERRA	≤ 45 kg (P2)
74-00179	16	10 mm	RIGHT	TERRA	≤ 45 kg (P2)
74-00185	17	10 mm	LEFT	TERRA	≤ 45 kg (P2)
74-00187	17	10 mm	RIGHT	TERRA	≤ 45 kg (P2)
74-00193	18	10 mm	LEFT	TERRA	≤ 60 kg (P3)
74-00195	18	10 mm	RIGHT	TERRA	≤ 60 kg (P3)
74-00201	19	10 mm	LEFT	TERRA	≤ 60 kg (P3)
74-00203	19	10 mm	RIGHT	TERRA	≤ 60 kg (P3)
74-00209	20	10 mm	LEFT	TERRA	≤ 60 kg (P3)
74-00211	20	10 mm	RIGHT	TERRA	≤ 60 kg (P3)
74-00217	21	10 mm	LEFT	TERRA	≤ 60 kg (P3)
74-00219	21	10 mm	RIGHT	TERRA	≤ 60 kg (P3)

TRANSTIBIAL ALIGNMENT

Transtibial Alignment Kit, Adult

FUNCTIONS AND BENEFITS

The Transtibial Alignment Kit, Adult is part of the Rehab'Impulse alignment system that permits prosthetic fitting for various levels of amputation and disarticulation of the lower extremities. The adult transtibial alignment kit consists of a set of components allowing different alignment configurations.

INTENDED USE

The Rehab'Impulse Transtibial Alignment Kit, Adult is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4). This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse prosthetic feet:

- Foot, SACH 2.0, sizes 22-28

INCLUDED IN DELIVERY*

- 1x Cup, TTA/TTC/TFC/LS
- 1x Convex disc
- 2x Concave cylinder, M10, adult
- 1x Convex ankle washer
- 1x Screw, countersunk head, FT, stainless steel, M10x60
- 1x Flat washer, stainless steel, 15/44/3

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

*Please note : Foot not included in the kit. To be ordered separately.



Article	Size	Side	Colour	User weight
74-00137	ADULT	RIGHT/LEFT	BEIGE	≤ 80 kg (P4)

Transtibial Alignment Kit, Adult, Long Stump

FUNCTIONS AND BENEFITS

The Transtibial Alignment Kit, Adult, Long Stump is part of the Rehab'Impulse alignment system that permits prosthetic fitting at various levels of amputation and disarticulation of the lower extremities. The adult long stump kit consists of a set of components allowing different alignment configurations intended for adults with a long residual limb.

INTENDED USE

The Rehab'Impulse Transtibial Alignment Kit, Adult, Long Stump is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4) and a long residual limb. This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse prosthetic feet:

- Foot, SACH 2.0, sizes 22-28

INCLUDED IN DELIVERY

- 1x Cup, TTA/TTC/TFC/LS
- 1x Convex disc
- 1x Concave ankle washer, adult
- 1x Flat washer, stainless steel, 15/44/3

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

OPTIONAL FOR FOOT FIXATION (NOT INCLUDED IN DELIVERY)

- 74-00880 Heavy Duty Nut Adult, For TT Long Stump (pack of 10)
- 74-01679 Nut, hexagonal M10, stainless steel, DIN 934
- 74-01688 Washer, Flat iron, zinc plated, diam. 10x25mm, thickness 2mm



Article	Size	Side	Colour	User weight
74-00229	ADULT	RIGHT/LEFT	BEIGE	≤ 80 kg (P4)

TRANSTIBIAL ALIGNMENT

Transtibial Alignment Kit, Child

FUNCTIONS AND BENEFITS

The Transtibial Alignment Kit, Child is part of the Rehab'Impulse alignment system that permits prosthetic fitting for various levels of amputation and disarticulation of the lower extremities. The child transtibial alignment kit consists of a set of components allowing different alignment configurations.

INTENDED USE

The Rehab'Impulse Transtibial Alignment Kit, Child is intended for prosthetic fitting of users with a body weight of up to 60 kg (P3). This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse prosthetic feet:

- Foot, SACH 2.0, sizes 14-21

INCLUDED IN DELIVERY*

- 1x Cup, TTA/TTC/TFC/LS
- 1x Convex disc
- 2x Concave cylinder, M8, child
- 1x Convex ankle washer
- 1x Screw, countersunk head, FT, stainless steel, M8x60
- 1x Flat washer, stainless steel, 10.5/40/1.5

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use: -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

*Please note: Foot not included in the kit. To be ordered separately.



Article	Size	Side	Colour	User weight
74-00221	CHILD	RIGHT/LEFT	BEIGE	≤ 60 kg (P3)

Transtibial Alignment Kit, Child, Long Stump

FUNCTIONS AND BENEFITS

The Transtibial Alignment Kit, Child, Long Stump is part of the Rehab'Impulse alignment system that permits prosthetic fitting at various levels of amputation and disarticulation of the lower extremities. The child long stump kit consists of a set of components allowing different alignment configurations intended for children with a long residual limb.

INTENDED USE

The Rehab'Impulse Transtibial Alignment Kit, Child, Long Stump is intended for prosthetic fitting of users with a body weight of up to 60 kg (P3) and a long residual limb. This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse prosthetic feet:

- Foot, SACH 2.0, sizes 14-21

INCLUDED IN DELIVERY

- 1x Cup, TTA/TTC/TFC/LS
- 1x Convex disc
- 1x Concave ankle washer, child
- 1x Flat washer, stainless steel, 15/44/3

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

OPTIONAL FOR FOOT FIXATION (NOT INCLUDED IN DELIVERY)

- 74-00879 Heavy Duty Nut Child, For TT Long Stump (pack of 10)
- Nut, hexagonal M8, HSS A2, DIN 934
- Washer, Flat iron, zinc plated, diam. 8x25mm, thickness 2mm



Article	Size	Side	Colour	User weight
74-00404	CHILD	RIGHT/LEFT	BEIGE	≤ 60 kg (P3)

TRANSTIBIAL ALIGNMENT

Heavy Duty Nut Adult, For TT Long Stump

FUNCTIONS AND BENEFITS

The Heavy Duty Nut Adult, For TT Long Stump is part of the Rehab'Impulse alignment system and is designed to provide compatibility with the Transtibial Alignment Kit, Adult, Long Stump and the Foot SACH 2.0 Adult. It is intended for adults and adolescents with a long residual limb.

INTENDED USE

The Heavy Duty Nut Adult, For TT Long Stump is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4). This product is not intended for excessive physical activities.

COMPATIBILITY

The heavy duty nut adult features an M10 female interface and is compatible with the Rehab'Impulse transtibial alignment kit adult and the SACH 2.0 prosthetic feet adult:

- 74-00229 Transtibial Alignment Kit, Adult, Long Stump
- Foot, SACH 2.0, sizes 22-28 (Adult)

INCLUDED IN DELIVERY

- 10x Heavy Duty Nut Child

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Material	Interface	User weight
74-00880	ADULT	STAINLESS STEEL	M10	≤ 80kg (P4)

Heavy Duty Nut Child, For TT Long Stump

FUNCTIONS AND BENEFITS

The Heavy Duty Nut Child, For TT Long Stump is part of the Rehab'Impulse alignment system and is designed to provide compatibility with the Transtibial Alignment Kit, Child, Long Stump and the Foot SACH 2.0 Child. It is intended for children with a long residual limb.

INTENDED USE

The Heavy Duty Nut Child, For TT Long Stump is intended for prosthetic fitting of users with a body weight of up to 60 kg (P3). This product is not intended for excessive physical activities.

COMPATIBILITY

The heavy duty nut child features an M8 female interface and is compatible with the Rehab'Impulse transtibial alignment kit child and the SACH 2.0 prosthetic feet child:

- 74-00404 Transtibial Alignment Kit, Child, Long Stump
- Foot, SACH 2.0, sizes 14-21 (Child)

INCLUDED IN DELIVERY

- 10x Heavy Duty Nut Child

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Material	Interface	User weight
74-00879	CHILD	STAINLESS STEEL	M8	≤ 60 kg (P3)

TRANSFEMORAL ALIGNMENT

Transfemoral Alignment Kit, Adult

FUNCTIONS AND BENEFITS

The Transfemoral Alignment Kit, Adult is part of the Rehab'Impulse alignment system that permits prosthetic fitting for various levels of amputation and disarticulation of the lower extremities. The adult transfemoral alignment kit consists of a set of components allowing different alignment configurations.

INTENDED USE

The Rehab'Impulse Transfemoral Alignment Kit, Adult is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4). This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse adult knee joints and prosthetic feet:

- 74-00243 Monocentric Knee Joint, Beige, Adult
- 74-00244 Monocentric Knee Joint, Olive, Adult
- 74-00245 Monocentric Knee Joint, Terra, Adult
- 74-00246 Polycentric Knee Joint, Beige, Adult
- 74-00247 Polycentric Knee Joint, Olive, Adult
- 74-00248 Polycentric Knee Joint, Terra, Adult
- Foot, SACH 2.0, sizes 22-28

INCLUDED IN DELIVERY*

- 1x Cup, transfemoral adult
- 1x Convex disc
- 1x Conical cup, transfemoral, adult
- 1x Concave cylinder, M10, adult
- 1x Convex ankle washer, adult
- 1x Screw, countersunk head, FT, stainless steel, M10x50
- 2x Flat washer, stainless steel, 15/55/3
- 1x Flat washer, stainless steel, 10.5/25/1.5
- 1x Hexagonal nut, stainless steel, M10
- 1x Coiled spring expansion pin, stainless steel, 5x45

*Please note: Knee joint and foot not included in the kit. To be ordered separately

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use: -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	Colour	User weight
74-00134	ADULT	RIGHT/LEFT	BEIGE	≤ 80 kg (P4)

Transfemoral Alignment Kit, Child

FUNCTIONS AND BENEFITS

The Transfemoral Alignment Kit, Child is part of the Rehab'Impulse alignment system that permits prosthetic fitting for various levels of amputation and disarticulation of the lower extremities. The child transfemoral alignment kit consists of a set of components allowing different alignment configurations.

INTENDED USE

The Rehab'Impulse Transfemoral Alignment Kit, Child is intended for prosthetic fitting of users with a body weight of up to 60 kg (P3). This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse children knee joints and prosthetic feet:

- 74-00240 Monocentric Knee Joint, Beige, Child
- 74-00241 Monocentric Knee Joint, Olive, Child
- 74-00242 Monocentric Knee Joint, Terra, Child
- Foot, SACH 2.0, sizes 14-21

INCLUDED IN DELIVERY*

- 1x Cup, TTA/TTC/TFC/LS
- 1x Convex disc
- 1x Conical cup, transfemoral, child
- 1x Concave cylinder, M8, child
- 1x Convex ankle washer, child
- 1x Screw, countersunk head, FT, stainless steel, M10x50
- 2x Flat washer, stainless steel, 15/44/3
- 1x Flat washer, stainless steel, 10.5/25/1.5
- 1x Hexagonal nut, stainless steel, M10
- 1x Coiled spring expansion pin, stainless steel, 5x40

*Please note: Knee joint and foot not included in the kit. To be ordered separately

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use: -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	Colour	User weight
74-00223	CHILD	RIGHT/LEFT	BEIGE	≤ 60 kg (P3)

TRANSFEMORAL ALIGNMENT

Transfemoral Alignment Kit, Child, Connector for Adult Foot

FUNCTIONS AND BENEFITS

The Transfemoral Alignment Kit, Child, Connector for Adult Foot is part of the Rehab'Impulse alignment system that permits prosthetic fitting for various levels of amputation for the lower extremities. The connector for adult foot offers the possibility to connect a children size knee joint to an adult foot SACH 2.0.

INTENDED USE

The Rehab'Impulse Transfemoral Alignment Kit, Child, Connector for Adult Foot is intended for prosthetic fitting of users with a body weight of up to 60 kg (P3). This product is not intended for excessive physical activities.

COMPATIBILITY

The components included in the set are compatible with the Rehab'Impulse children knee joints and adult prosthetic feet:

- 74-00240 Monocentric Knee Joint, Beige, Child
- 74-00241 Monocentric Knee Joint, Olive, Child
- 74-00242 Monocentric Knee Joint, Terra, Child
- Foot, SACH 2.0, sizes 22-28

INCLUDED IN DELIVERY

- 1x Concave cylinder child, connector for adult foot
- 1x Coiled spring expansion pin, stainless steel, 5x40

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	Colour	User weight
74-00227	CHILD	RIGHT/LEFT	GREY	≤ 60 kg (P3)

KNEE JOINTS

Monocentric Knee Joint, Adult

FUNCTIONS AND BENEFITS

The Monocentric Knee Joint, Adult is a durable and versatile prosthetic knee joint with an integrated manual locking system. This monocentric knee joint provides a bending angle of up to 125°.

INTENDED USE

The Rehab'Impulse Monocentric Knee Joint, Adult is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4). This product is not intended for excessive physical activities.

COMPATIBILITY

The monocentric knee joint is compatible with the Rehab'Impulse alignment system. A specially designed pyramid adapter offers compatibility with standard modular tube clamp adapters.

- 74-00134 Transfemoral Alignment Kit, Adult
- 74-00337 Pyramid Adapter, For Knee Joint Adult

INCLUDED IN DELIVERY

- 1x Monocentric Knee Joint, Adult


STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

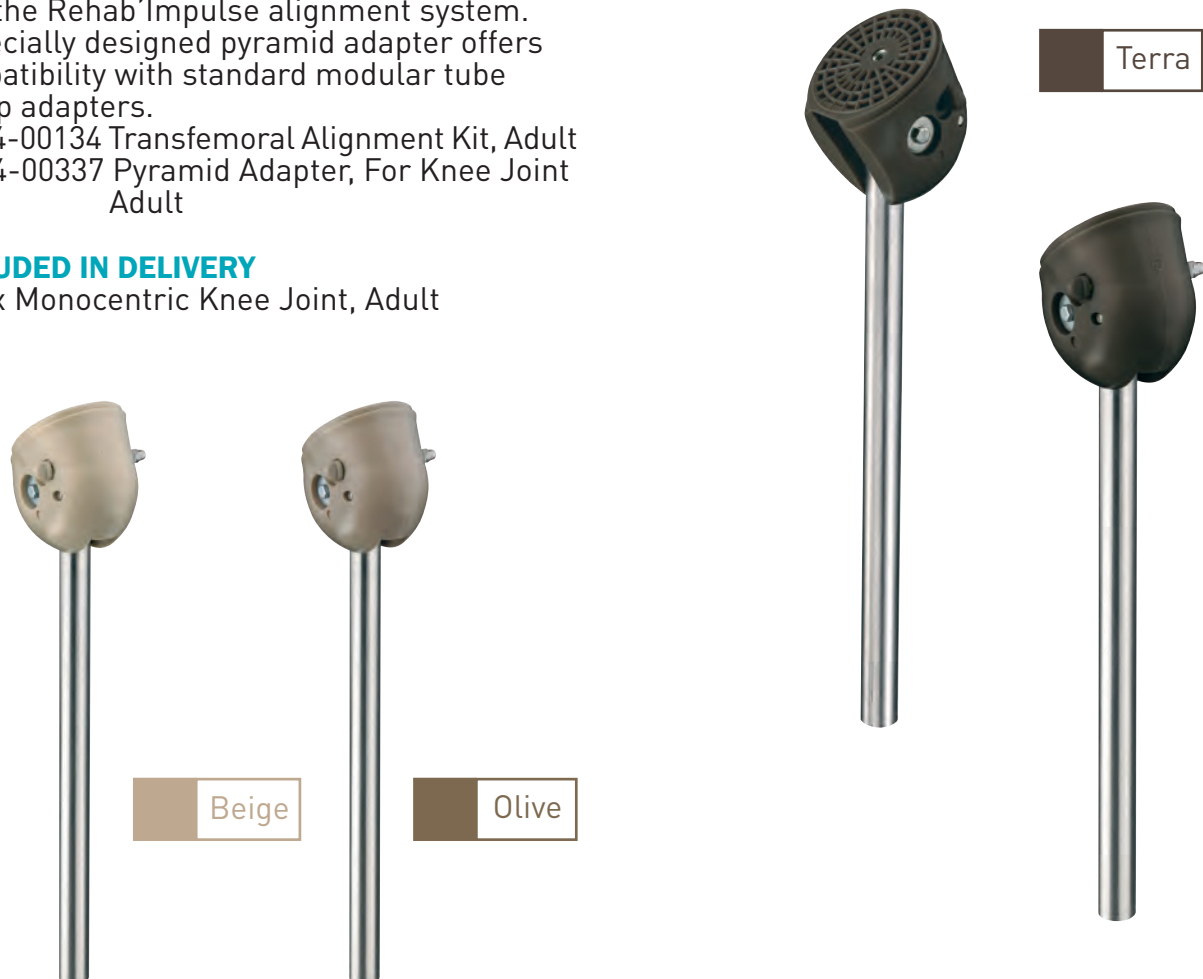

Article	Size	Side	Colour	User weight
74-00243	ADULT	RIGHT/LEFT	BEIGE	≤ 80 kg (P4)
74-00244	ADULT	RIGHT/LEFT	OLIVE	≤ 80 kg (P4)
74-00245	ADULT	RIGHT/LEFT	TERRA	≤ 80 kg (P4)

KNEE JOINTS

Bumper Kit Adult, For Monocentric Knee Joint

FUNCTIONS AND BENEFITS

The Bumper Kit Adult, For Monocentric Knee Joint is a spare part to the Rehab'Impulse Monocentric Knee Joint, Adult.

INTENDED USE

The Bumper Kit Adult, For Monocentric Knee Joint is intended for replacement of worn out parts on the monocentric knee joint adult. It consists of a rubber bumper and a corresponding M5 screw for the fixation on the knee joint.

COMPATIBILITY

The bumper kit adult is compatible with the Rehab'Impulse monocentric knee joint adult:

- 74-00243 Monocentric Knee Joint, Beige, Adult
- 74-00244 Monocentric Knee Joint, Olive, Adult
- 74-00245 Monocentric Knee Joint, Terra, Adult

INCLUDED IN DELIVERY

- 10x Rubber bumper
- 10x Screw M5, stainless steel

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Material Bumper	Material Screw	User weight
74-00780	ADULT	TPE	STAINLESS STEEL	≤ 80 kg (P4)

KNEE JOINTS

Monocentric Knee Joint, Child

FUNCTIONS AND BENEFITS

The Monocentric Knee Joint, Child is a durable and versatile prosthetic knee joint with an integrated manual locking system. This monocentric knee joint provides a bending angle of up to 125°.

INTENDED USE

The Rehab'Impulse Monocentric Knee Joint, Child is intended for prosthetic fitting of users with a body weight of up to 60 kg (P3). This product is not intended for excessive physical activities.

COMPATIBILITY

The monocentric knee joint is compatible with the Rehab'Impulse alignment system:

- 74-00223 Transfemoral Alignment Kit, Child

INCLUDED IN DELIVERY

- 1x Monocentric Knee Joint, Child

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	Colour	User weight
74-00240	CHILD	RIGHT/LEFT	BEIGE	≤ 60 kg (P3)
74-00241	CHILD	RIGHT/LEFT	OLIVE	≤ 60 kg (P3)
74-00242	CHILD	RIGHT/LEFT	TERRA	≤ 60 kg (P3)

KNEE JOINTS

Bumper Kit Child, For Monocentric Knee Joint

FUNCTIONS AND BENEFITS

The Bumper Kit Child, For Monocentric Knee Joint is a spare part to the Rehab'Impulse Monocentric Knee Joint, Child.

INTENDED USE

The Bumper Kit Child, For Monocentric Knee Joint is intended for replacement of worn out parts on the monocentric knee joint child. It consists of a rubber bumper and a corresponding M4 screw for the fixation on the knee joint.

COMPATIBILITY

The bumper kit child is compatible with the Rehab'Impulse monocentric knee joint child:

- 74-00240 Monocentric Knee Joint, Beige, Child
- 74-00241 Monocentric Knee Joint, Olive, Child
- 74-00242 Monocentric Knee Joint, Terra, Child

INCLUDED IN DELIVERY

- 10x Rubber bumper
- 10x Screw M4, stainless steel

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Material Bumper	Material Screw	User weight
74-00781	CHILD	TPE	STAINLESS STEEL	≤ 60 kg (P3)

KNEE JOINTS

Locking Spring Kit, For Monocentric Knee Joint, Adult/Child

FUNCTIONS AND BENEFITS

The Locking Spring Kit, For Monocentric Knee Joint, Adult/Child offers the possibility to equip a monocentric knee joint with a locking spring. This locking spring activates the locking lever when the knee joint is in full extension.

INTENDED USE

The Locking Spring Kit, For Monocentric Knee Joint, Adult/Child is intended to be used with the Rehab'Impulse monocentric knee joints adult or child.

COMPATIBILITY

The locking spring as well as the template and the tools contained in the kit are compatible with the Rehab'Impulse monocentric knee joints:

- 74-00243 Monocentric Knee Joint, Beige, Adult
- 74-00244 Monocentric Knee Joint, Olive, Adult
- 74-00245 Monocentric Knee Joint, Terra, Adult

- 74-00240 Monocentric Knee Joint, Beige, Child
- 74-00241 Monocentric Knee Joint, Olive, Child
- 74-00242 Monocentric Knee Joint, Terra, Child

INCLUDED IN DELIVERY

- 50x Locking spring
- 2x Mounting hook
- 1x Drill template
- 2x Drill 4mm

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	User weight
74-00789	ADULT/CHILD	RIGHT/LEFT	≤ 80 kg (P4)

KNEE JOINTS

Polycentric Knee Joint, Adult

FUNCTIONS AND BENEFITS

The Polycentric Knee Joint, Adult is designed to provide improved stability throughout stance phase and smooth progression during swing phase with increased toe-clearance. This polycentric knee joint with extension assist provides a wide bending angle of up to 165° with increased stability when kneeling.

INTENDED USE

The Rehab'Impulse Polycentric Knee Joint, Adult is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4). This product is not intended for excessive physical activities.

COMPATIBILITY

The polycentric knee joint is compatible with the Rehab'Impulse alignment system. A specially designed pyramid adapter offers compatibility with standard modular tube clamp adapters.

- 74-00134 Transfemoral Alignment Kit, Adult
- 74-00337 Pyramid Adapter, For Knee Joint Adult

INCLUDED IN DELIVERY

- 1x Polycentric Knee Joint, Adult3x
Spacer for stability adjustment (see user manual for mounting instructions)

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

Beige



Olive

Terra



Article	Size	Side	Colour	User weight
74-00246	ADULT	RIGHT/LEFT	BEIGE	≤ 80 kg (P4)
74-00247	ADULT	RIGHT/LEFT	OLIVE	≤ 80 kg (P4)
74-00248	ADULT	RIGHT/LEFT	TERRA	≤ 80 kg (P4)

KNEE JOINTS

Pyramid Adapter, For Knee Joint Adult

FUNCTIONS AND BENEFITS

The Pyramid Adapter, For Knee Joint Adult is part of the Rehab'Impulse alignment system and is designed to provide compatibility with standard modular tube clamp adapters used as interface with common modular hip joints.

INTENDED USE

The Rehab'Impulse Pyramid Adapter, For Knee Joint Adult is intended for prosthetic fitting of users with a body weight of up to 80 kg (P4). This product is not intended for excessive physical activities.

COMPATIBILITY

- 74-00243 Monocentric Knee Joint, Beige, Adult
- 74-00244 Monocentric Knee Joint, Olive, Adult
- 74-00245 Monocentric Knee Joint, Terra, Adult
- 74-00246 Polycentric Knee Joint, Beige, Adult
- 74-00247 Polycentric Knee Joint, Olive, Adult
- 74-00248 Polycentric Knee Joint, Terra, Adult

INCLUDED IN DELIVERY

- 1x Pyramid
- 1x Threaded pin M8/M10
- 1x Hexagonal Nut, stainless steel, M8

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	Material	User weight
74-0337	ADULT	RIGHT/LEFT	STAINLESS STEEL	≤ 80 kg (P4)

KNEE JOINTS

Maintenance Kit, For Polycentric Knee Joint

FUNCTIONS AND BENEFITS

The Maintenance Kit, For Polycentric Knee Joint offers a selection of the most commonly used spare parts for the Rehab'Impulse Polycentric Knee Joint, Adult.

INTENDED USE

The Maintenance Kit, For Polycentric Knee Joint allows replacing worn out parts on polycentric knee joints or to change the extension assist spring.

COMPATIBILITY

The articles contained in this kit are compatible with the Rehab'Impulse polycentric knee joints:

- 74-00246 Polycentric Knee Joint, Beige, Adult
- 74-00247 Polycentric Knee Joint, Olive, Adult
- 74-00248 Polycentric Knee Joint, Terra, Adult

INCLUDED IN DELIVERY

- 25x Spring support
- 25xBumper
- 50xScrew for bumper
- 100x Screw M6 (lateral linkage)
- 100x Washer M6
- 100x Screw M5 (central linkage)
- 25x Pin D5 (central linkage)
- 50xBushing for spring piston
- 100x Bushing front linkage
- 100x Bushing rear linkage
- 25x Extension spring low
- 25x Extension spring hard
- 100x Screw M4 (friction adjustment)
- 100x Washer M4, large
- 50x Spacer for bumper
- 1x Glue for friction adjustment screw

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Weight
74-01341	4.15 kg

UPPER LIMBS

Passive Hand, Large

FUNCTIONS AND BENEFITS

The Rehab'Impulse Passive Hand is a cosmetic hand made of soft thermoplastic polyurethane. Besides its natural appearance, it allows passive handling and holding of small objects. It is available in two sizes and four colours. The large size is recommended for men while the medium size is recommended for women and adolescents.

Both hand sizes feature an M10 male interface.

INTENDED USE

This product is intended for prosthetic fitting on adults and adolescents and is exchangeable with a hook or a working ring.

COMPATIBILITY

The passive hands are compatible with the Rehab'Impulse elbow joint and the Rehab'Impulse wrist unit.

- 74-00239 Prosthetic Elbow Joint, Beige, Adult
- 74-00238 Prosthetic Elbow Joint, Olive, Adult
- 74-00237 Prosthetic Elbow Joint, Terra, Adult
- 74-00449 Wrist Unit Kit, Beige
- 74-00450 Wrist Unit Kit, Olive
- 74-00451 Wrist Unit Kit, Terra



INCLUDED IN DELIVERY

- 1x Passive Hand, Large

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

Article	Size	Side	Colour	Wrist interface
74-00063	L	LEFT	BEIGE	M10
74-00065	L	RIGHT	BEIGE	M10
74-00051	L	LEFT	DARK BEIGE	M10
74-00053	L	RIGHT	DARK BEIGE	M10
74-00067	L	LEFT	OLIVE	M10
74-00069	L	RIGHT	OLIVE	M10
74-00059	L	LEFT	TERRA	M10
74-00061	L	RIGHT	TERRA	M10

UPPER LIMBS

Passive Hand, Medium

FUNCTIONS AND BENEFITS

The Rehab'Impulse Passive Hand is a cosmetic hand made of soft thermoplastic polyurethane. Besides its natural appearance, it allows passive handling and holding of small objects. It is available in two sizes and four colours. The large size is recommended for men while the medium size is recommended for women and adolescents.

Both hand sizes feature an M10 male interface.

INTENDED USE

This product is intended for prosthetic fitting on adults and adolescents and is exchangeable with a hook or a working ring.

COMPATIBILITY

The passive hands are compatible with the Rehab'Impulse elbow joint and the Rehab'Impulse wrist unit.

- 74-00239 Prosthetic Elbow Joint, Beige, Adult
- 74-00238 Prosthetic Elbow Joint, Olive, Adult
- 74-00237 Prosthetic Elbow Joint, Terra, Adult
- 74-00449 Wrist Unit Kit, Beige
- 74-00450 Wrist Unit Kit, Olive
- 74-00451 Wrist Unit Kit, Terra

INCLUDED IN DELIVERY

- 1x Passive Hand, Médium

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine



Article	Size	Side	Colour	Wrist interface
74-00071	M	LEFT	BEIGE	M10
74-00073	M	RIGHT	BEIGE	M10
74-00055	M	LEFT	DARK BEIGE	M10
74-00057	M	RIGHT	DARK BEIGE	M10
74-00075	M	LEFT	OLIVE	M10
74-00077	M	RIGHT	OLIVE	M10
74-00079	M	LEFT	TERRA	M10
74-00081	M	RIGHT	TERRA	M10

UPPER LIMBS

Prosthetic Elbow Joint

FUNCTIONS AND BENEFITS

The Rehab'Impulse Prosthetic Elbow Joint is a durable and versatile prosthetic elbow joint with an integrated manual locking system allowing the user to set the angle of the elbow (7 positions). The Rehab'Impulse wrist adapter included with the elbow, is designed to provide compatibility of the Rehab'Impulse Elbow Joint with the Rehab'Impulse Passive Hand or a commercially available hook.

INTENDED USE

This product is intended for prosthetic fitting on adults. This product is not intended for excessive physical activities.

COMPATIBILITY

The elbow joint and its wrist unit is compatible with the Rehab'Impulse passive hand.

- Passive hand, Size L
- Passive hand, Size M

INCLUDED IN DELIVERY

- 1x Elbow Joint
- 1x Wrist unit kit (1x wrist unit / 4x screw, stainless steel 3x10mm)


STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS

Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine

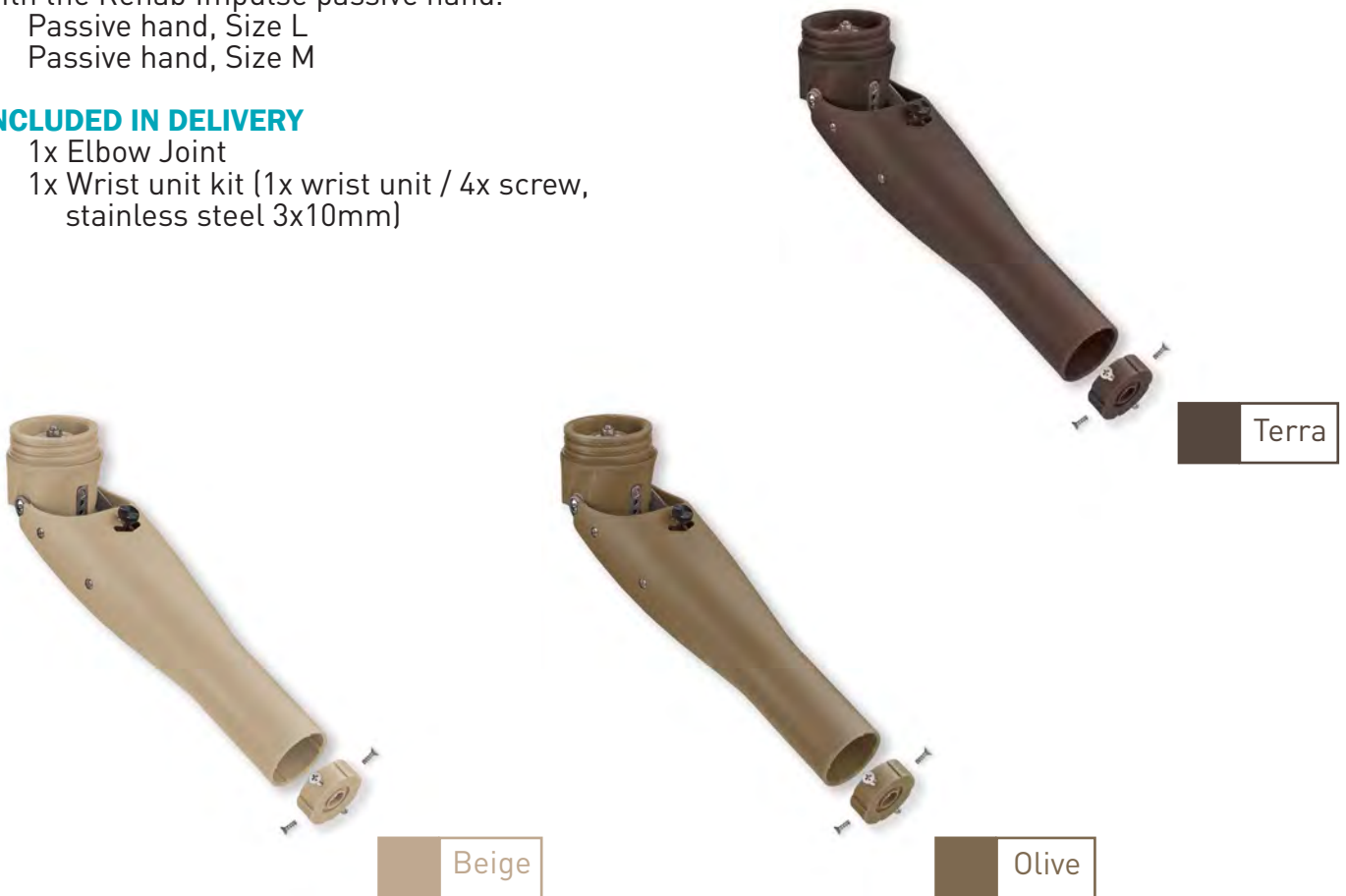

Article	Size	Side	Colour	Wrist interface
74-00239	ADULT	RIGHT/LEFT	BEIGE	M10
74-00238	ADULT	RIGHT/LEFT	OLIVE	M10
74-00237	ADULT	RIGHT/LEFT	TERRA	M10

Wrist Unit Kit

FUNCTIONS AND BENEFITS

The Rehab'Impulse Wrist Unit Kit is designed to provide compatibility of the Rehab'Impulse Elbow Joint with the Rehab'Impulse Passive Hand or a commercially available hook. It can also be mounted to a transradial prosthetic device.

INTENDED USE

This product is intended for prosthetic fitting on adults. This product is not intended for excessive physical activities.

COMPATIBILITY

The wrist unit is compatible with the Rehab'Impulse passive hand.

- Passive hand, Size L
- Passive hand, Size M

INCLUDED IN DELIVERY

- 1x Wrist unit
- 4x Screw, stainless steel 3x10mm

STORAGE CONDITIONS

- Store product in dry conditions
- Store at room temperature (ideally between 15°C and 25°C)
- Keep out of direct sunlight or other sources of light with a high UV content
- Take precautionary measures against sparking and fire
- Keep in original packaging before use

ENVIRONMENTAL CONDITIONS












Recommended environmental conditions:

- Temperature range for use : -10°C to 60°C (14°F to 140°F)
- Relative humidity 0% to 90%, no condensing situation
- Avoid exposure to dust, sand, salt water, acids and urine












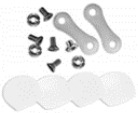

Article	Size	Side	Colour	Wrist interface
74-00449	ADULT	RIGHT/LEFT	BEIGE	M10
74-00450	ADULT	RIGHT/LEFT	OLIVE	M10
74-00451	ADULT	RIGHT/LEFT	TERRA	M10











COMPONENTS

Image for illustration only Original item may vary	Reference	Description
	74-02075	Rivet Kit For Belt and Polypropylene Cosmetic Cover Beige (20pcs D6mm + 20pcs D8mm + 5pairs belt rivet)
	74-02076	Rivet Kit For Belt and Polypropylene Cosmetic Cover Olive (20pcs D6mm + 20pcs D8mm + 5pairs belt rivet)
	74-02077	Rivet Kit For Belt and Polypropylene Cosmetic Cover Terra (20pcs D6mm + 20pcs D8mm + 5pairs belt rivet)
	74-01052	Silicone Valve and Removable Valve Seat - Set
	74-01059	FOOT, ankle flexure joint CAP set, L, 10 ankle flex, pairs
	74-01060	FOOT, ankle flexure joint CAP set, M, 10 ankle flex, pairs
	74-01061	FOOT, ankle flexure joint CAP set, pediatric, 10 ankle flex, pairs
	74-01062	FOOT, ankle flexure dorsi assist 85, L, 10 ankle flex + dummy
	74-01063	FOOT, ankle flexure dorsi assist 85, M, 10 ankle flex + dummy
	74-01064	FOOT, ankle flexure dorsi assist 85, pediatric, 10 ankle flex + dummy
	74-01065	FOOT, ankle flexure joint, L, 10 ankle flex + dummy, pairs

COMPONENTS

Image for illustration only Original item may vary	Reference	Description
	74-01066	FOOT, ankle flexure joint, M, 10 ankle flex + dummy, pairs
	74-01068	FOOT, ankle flexure joint, pediatric, 10 ankle flex + dummy, pairs
	74-01069	Tamarack Flexure Joint Molding Dummies, large, pair
	74-01070	Tamarack Flexure Joint Molding Dummies, medium, pair
	74-01071	Tamarack Flexure Joint Molding Dummies, pediatric, pair
	74-01072	Tamarack Plantar Flexion Limiter Kit, large
	74-01073	Tamarack Plantar Flexion Limiter Kit, medium
	74-01368	Orthotic Knee Joints, Drop lock 1001, adult, 5/32in x 3/4in
	74-01369	Orthotic Knee Joints, Drop lock 1001, child, 5/32in x 5x8in
	On request	HARNESS, for upper limb prosthesis, cable, hose, connectors
	74-01240	ALUMINIUM SEAT VALVE, for AK prosthese + thermoforming dummy



COMPONENTS

Image for illustration only Original item may vary	Reference	Description
	74-01241	HIP JOINT, adaptor tube, tilt 20 degree
	On request	HIP JOINT, for prosthese
	On request	FOOT ADAPTER M10 with bolted connection
	On request	FOOT ADAPTER M8 with bolted connection
	74-01858	HOOK, stainless steel, adult, left
	74-01059	HOOK, stainless steel, adult, right
	74-01239	FORMING TOOL, aluminium, for seat valve OSBOVSB0041
	74-01753	SOCKETS BRIMS IPOS BK, set of brims for casting
	74-01323	SOCKETS BRIMS IPOS AK, set of brims for casting
	74-00063	PEA Care 2.0 size 1

COMPONENTS

Image for illustration only Original item may vary	Reference	Description
	74-01393	PEA Care 2.0 size 2
	74-01390	Clubycare boot with laces, Size 24, pair
	74-01392	Clubycare boot with laces, Size 26, pair
	74-01394	Clubycare boot with laces, Size 28, pair
	74-01396	Clubycare boot with laces, Size 30, pair
	74-01398	Clubycare boot with velcro, Size 14, pair
	74-01400	Clubycare boot with velcro, Size 16, pair
	74-01402	Clubycare boot with velcro, Size 18, pair
	74-01404	Clubycare boot with velcro, Size 20, pair
	74-01406	Clubycare boot with velcro, Size 22, pair

RAW MATERIALS

Image for illustration only Original item may vary	Reference	Description
	74-01079	POLYPROPYLENE WELDING ROD, 4mm, beige colour, roll
	74-01080	POLYPROPYLENE WELDING ROD, 4mm, olive colour, roll
	74-01081	POLYPROPYLENE WELDING ROD, 4mm, terra colour, roll
	74-01082	HOMOPOLYMER, 3mm x 1m x 2m, 5.6Kg, terra colour
	74-01083	HOMOPOLYMER, 3mm x 1m x 2m, 5.5Kg, beige colour
	74-01084	HOMOPOLYMER, 4mm x 1m x 2m, 7.5Kg, beige colour
	74-01085	HOMOPOLYMER, 5mm x 1m x 2m, 9.2Kg, beige colour
	74-01086	HOMOPOLYMER, 3mm x 1m x 2m, 5.5Kg, olive colour
	74-01087	HOMOPOLYMER, 4mm x 1m x 2m, 7.5Kg, olive colour
	74-01088	HOMOPOLYMER, 5mm x 1m x 2m, 9.2Kg, olive colour












RAW MATERIALS

Image for illustration only Original item may vary	Reference	Description
	74-01089	HOMOPOLYMER, 4mm x 1m x 2m. 7.5Kg, terra colour
	74-01090	HOMOPOLYMER, 5mm x 1m x 2m, 9.2Kg, terra colour
	74-00825	EVA FOAM, 3mm x 0.95m x 0.95m, 0.90m ² , beige
	74-00826	EVA FOAM, 6mm x 0.95m x 0.95m, 0.90m ² , beige
	74-00827	EVA FOAM, 12mm x 1.10m x 1.10m, 1.21m ² , beige
	74-00828	EVA FOAM, 3mm x 0.95m x 0.95m, 0.90m ² , olive
	74-00829	EVA FOAM, 6mm x 0.95m x 0.95m, 0.90m ² , olive
	74-00830	EVA FOAM, 12mm x 1.10m x 1.10m, 1.21m ² , olive
	74-00831	EVA FOAM, 3mm x 0.95m x 0.95m, 0.90m ² , terra brown
	74-00832	EVA FOAM, 6mm x 0.95m x 0.95m, 0.90m ² , terra brown
	74-00833	EVA FOAM, 12mm x 1.10m x 1.10m, 1.21m ² , terra brown


















HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01002	RIVETING BAR, with rivet hole, punched rivet anvil, 680mm
	74-01842	Handle with 3/8" connection
	74-01843	Extension with 3/8" connection L=508mm
	74-01844	Insert Hex with 3/8" connection, 5mm
	74-01845	Insert Hex with 3/8" connection, 6mm
	74-02054	Insert Hex with 3/8" connection, 5mm, long, with ball end
	74-02055	Insert Hex with 3/8" connection, 6mm, long, with ball end
	74-01183	CHISEL SET, for wood, 5 pcs., plastic handle 6/12/16/20/25mm
	74-01200	HACKSAW, tubular frame, protected handle, 300mm
	74-01201	HAMMER, adaptable nylon tip mallet, d. 40-45mm, 630g
	74-01202	KEY SET, hexagonal male, "Allen", 1.5 to 10mm, spherical end
	74-01204	KNIFE/CUTTER, retract. chang. snap-off blade, 165mm, plastic
	74-01206	PLASTER MOULDING SPATULA, "HSS Butter Knife"













HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01208	RIVETING GUN, + 450 rivets, d. 4 to 4.8mm, thick 1.3 to 13mm
	74-01212	SCREWDRIVER, flat head 3.5mm x 75mm blade, No.1
	74-01213	SCREWDRIVER, flat head 5.5mm x 100mm blade, No.3
	74-01214	SCREWDRIVER, flat head, 8 x 160mm blade, No.5
	74-01211	SCREWDRIVER, flat head 13mm x 200mm blade, No.7
	74-01215	SCREWDRIVER, Phillips head n.0 x 60mm blade
	74-01216	SCREWDRIVER, Phillips head n.1 x 80mm blade
	74-01704	SCREWDRIVER, Phillips head n.2 x 100mm blade
	74-01705	SCREWDRIVER, Phillips head n.3 x 150mm blade
	74-01217	SOCKET AND BIT, metric, set 1/2" from 10 to 32mm
	74-01218	SPANNER SET, 15deg offset ring, 8 pces 6mm to 22mm
	74-01219	SPANNER, combinaison, open + 15deg angled ring, 10mm
	74-01220	SPANNER, combinaison, open + 15deg angled ring, 17mm
	74-01221	TOOL CASE, Tool-Craft, 127 pcs. 1/4" +1/2", chrome vanadium
	74-01225	WRENCH, OPEN END, SET, 12 pces from 6 to 32mm
	74-01202	KEY SET, hexagonal male, "Allen", 1.5 to 10mm, spherical end











HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01321	SOCKET AND BIT, DynaTorqueH2-6
	74-01552	WIRE BRUSH, for welder
	74-01554	HAMMER, for welder
 	74-01698	FILE triangular coarse cut 250 mm
 	74-01701	FILE triangular medium cut 250 mm
 	74-01702	FILE square coarse cut 10 x 250 mm
 	74-01703	FILE square medium cut 250 mm
	74-01706	KEY, hexagonal male, "Allen", 6mm with spherical end
	74-01709	SPANNER, combinaison, open + 15deg angled ring, 11mm
	74-01710	SPANNER, combinaison, open + 15deg angled ring, 13mm
	74-01711	SPANNER, combinaison, open + 15deg angled ring, 19mm
	74-01712	SPANNER, combinaison, open + 15deg angled ring, 21mm
	74-01713	HAND SHEARS, straight, for metal sheet, semi-hard 1,2mm max., 260mm











HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01714	PUNCH and CHIESEL set of 6 pcs.
	74-01715	PUNCH LETTERS, 26 letters, size 6mm, plus dot, in PP box
	74-01716	PUNCH NUMBERS, 9 numbers, size 4mm, in PP box
	74-01717	HOLE PUNCH, 2mm
	74-01718	HOLE PUNCH, 3mm
	74-01719	HOLE PUNCH, 4mm
	74-01720	HOLE PUNCH, 5mm
	74-01721	HOLE PUNCH, 6mm
	74-01722	HOLE PUNCH, 8mm
	74-01723	HOLE PUNCH, 10mm
	74-01724	HOLE PUNCH, 12mm
	74-01734	DIE STOCK, for dies M3 / M4, d.20 X 5mm th.











HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01735	DIE STOCK, for dies M5 / M6, diam 20 X 7mm th.
	74-01736	DIE STOCK, for dies M8, diam 25 X 9mm th.
	74-01737	DIE STOCK, for dies M10, diam 30 X 11mm th.
	74-01738	DIE STOCK, for dies M12, diam 38 X 14mm th.
	74-01739	TAP WRENCH, for metrique tap M3 - M12
	74-01748	SCRIBER, steel, hardened chrome-vanadium tips, 190mm
	74-01749	CALIPER GAUGE, max. 150mm, universal, to 1/10mm
	74-01750	Transfemoral Measuring Gauge ML (ICS)
	74-01752	MEASURING INSTRUMENT, for inside circumference, 24 - 66cm
	74-01755	FILE WITH HANDLE, "Stanley Surform", round, blade 250mm

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01756	BAR CLAMP, max. opening 250mm, bar 30 x 8mm
	74-01757	BAR CLAMP, max. opening 400mm, bar 30 x 8mm
	74-01758	BAR CLAMP, max. opening 800mm, bar 35 x 9mm
	74-01803	VICE, BENCH, jaws 140mm, open. 200mm, without swivel base
	74-01805	TRIPOD, for shoes, shoemaker tool
	74-01806	ANVIL, special for embossing work made of cast steel
	74-01807	PLIER, CRIMPING, for all terminal on cables 0.5 to 6mm ²
	74-01812	SCREWDRIVER, TESTER, 110-250V, with clip
	On request	SCISSOR, plaster cast shears, toothed, stainless st., 210mm
	On request	SET, PLASTER CASTS REMOVAL, instruments
	On request	SCISSORS, PLASTER SHEARS, STILLE, 37 cm, reinforced blade

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	On request	KNIFE, PLASTER, ESMACH, 18 cm
	On request	SPREADER, PLASTER, HENNIG, 27 cm
	On request	BREAKER, PLASTER CAST, WOLFF-BOEHLER, 18 cm
	74-01226	Goniometre, long branches
	74-00683	LEVEL, aluminium, 400 mm, standard, vertic. horizont. vials
	74-00684	ANVIL, with 2 horns, 20kg
	74-00687	HOLE PUNCH, arch, 15mm
	74-00688	HOLE PUNCH, arch, 20mm
	74-00689	HOLE PUNCH, arch, 40mm
	74-00690	HAMMER, ball pen, 250g, wooden handle










HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00691	HAMMER, ball pen, 400g, wooden handle
	74-00692	SCISSOR, for bandages, "Lister"
	74-00694	SHARPENING STONE, fine-medium, 200 x 50 x25mm
	74-00709	CUTTING DEVICE, steel shears for 5mm sheet, 11mm round bar
	74-00710	DIRECT READING CALIPER for medio-lateral-antero-post measure
	74-00711	DRAW KNIFE, 250mm, 2 wood handle
	74-00718	WIRE BRUSH, for file cleaning, 20 x 40 x 230mm
	74-00720	FILE, flat, coarse cut, 25 x 250 mm
	74-00729	FILE, flat, medium cut, 25 x 250mm
	74-00730	FILE, half-round, coarse cut, 25 x 250mm











HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00731	FILE, half-round, medium cut, 25 x 250 mm
	74-00732	FILE, round, coarse cut, 10 x 250mm
	74-00733	FILE, round, medium cut, 10 x 250mm
	74-00734	HAMMER, standard, hardened steel, wood handle, 200g
	74-00736	HAMMER, riveting, 500 gr
	74-00742	GONIOMETER
	74-00743	LEVEL, for pelvis
	74-00747	HAMMER, rubber mallet, d. 65mm, 520g
	74-00778	PUNCH LETTERS, 26 letters, size 4mm, plus dot, in PP box
	74-00779	PLIER, long nose, 200mm

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00785	PLIER, multigrip, standard, max. open. 52mm, 250mm length
	74-00786	SCISSORS, office 21cm, s-steel, for right and left-handed
	74-00787	PUNCH NUMBERS, 9 numbers, size 6mm, in PP box
	74-00791	PUNCH SET, drift punch for spring pin, d.2,3,4,5,6,8mm
	74-00798	PLASTIC HANDLES, for files 250mm, L.121mm
	74-00802	PLUMB BOB, plumb line with clearance plate, with 4m line 2mm
	74-00803	PNEUMATIC CHIPPING HAMMER + set of chisels
	74-00806	Cast scissor "Esmarch" 220 mm
	74-00824	SCISSORS, universal, 18cm, toothed edges, for bandage-cloth
	74-00808	CONTOURING INSTRUMENT, round beak, 4 + 6mm, 265mm











HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00809	CONTOURING INSTRUMENT, round beak, 6 + 8mm, 265mm
	74-00820	CONTOURING INSTRUMENT, round beak, 7 + 9mm, 265mm
	74-00821	CONTOURING INSTRUMENT, square beak, 4+6mm, 250mm
	74-00822	CONTOURING INSTRUMENT, square beak, 6+8mm, 250mm
	74-00823	CONTOURING INSTRUMENT, square beak, 7+9mm, 250mm
	74-00834	CONTOURING INSTRUMENT, flat contouring, 500mm
	74-00835	Foot measuring device "heider"
	74-00839	PUNCH, REVOLVING, hole punch plier, with 6 tips up 2 to 5mm
	74-00840	RIVET SETTER KIT, steel, diam. 2 to 6 mm (5 pcs) diam. rivet
	74-00841	SCISSORS, PLASTER SHEARS, BRUNS, 24cm










HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00842	GLUE CONTAINER 0.9L, EMPTY
	74-00843	GLUE CONTAINER 1.5L, EMPTY
	74-00845	RIVET HEADER KIT, steel, diam. 2 to 6 mm (5 pcs) diam. rivet
	74-00847	PUNCH, center punch, chrome finish, 120mm length
	74-00853	PLASTER MIXING, bowl, 0.50l
	74-00866	AWL, piercing, round, d. 6mm, blade 100mm, plastic handle
	74-00869	KNIFE, for leather, "Tina" wide blade, L. 240mm
	74-00870	PLASTER MOULDING SPATULA, "double end"
	74-00871	PLIER, universal, 160/5mm, electrician
	74-00872	PLIER, LOCK GRIP, 100mm max. opening, 250mm length

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00873	PLASTER MOULDING TOOL, set of 4 pieces
	74-00874	DEBURRING TOOL, changeable blade, plastic handle
	74-00877	RULE, stainless, flexible, 0.5mm, 2 sides mm-1/2mm, 300mm
	74-00878	RULE, stainless, flexible, 0.5mm, 2 sides mm-1/2mm, 500mm
	74-00881	SQUARE, length 150mm x height 100mm, stainless steel
	74-00882	MEASURING TAPE, 2m, 13mm tape, metric and inch, rolling case
	74-00884	CALIPER, BODY, gauge, 600mm, for measuring body segment
	74-00887	CALIPER GAUGE, max. 250mm, long jaws, + inside measure
	74-00888	FILE WITH HANDLE, "Stanley Surform", standard, flat blade 250mm
	74-00889	MEASURING TAPE, tailor type, flexible, 1.5m

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00886	RIVET HEADER, steel, burnished, for 3mm diameter rivets
	74-00890	RIVET HEADER, steel, burnished, for 4mm diameter rivets
	74-00891	RIVET HEADER, steel, burnished, for 5mm diameter rivets
	74-00892	RIVET SETTER, steel, coated, for 3mm diameter rivets
	74-00893	RIVET SETTER, steel, coated, for 4mm diameter rivets
	74-00894	RIVET SETTER, steel, coated, for 5mm diameter rivets
	74-00896	PLIER, side cutting, 160mm
	74-00897	(glue container) SPARE BRUSH for 0.45L
	74-00899	(glue container) SPARE BRUSH for 0.9 & 1.5L
	74-00900	GLUE CONTAINER 0.45L, EMPTY

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-00902	Trimming scissor PVC-coated handles fine-toothed, 210mm
	74-00910	PLIER, double action cutting, 180mm
	74-00911	SCISSOR, trimming scissor, "tailor", 235mm
	74-00912	PIPE CUTTER, heavy duty, for steel pipes 1/8" to 2"
	74-00915	FILE NEEDLE SET, 6 pieces warding files 160mm
	74-00918	WIRE BRUSH, Stainless steel wires, 3 rows
	74-01091	HOLE PUNCH, arch, 30mm
	74-01423	Punch set for tubular rivets 74-00914
	74-01424	Punch sset, tubular rivets 74-00913
	74-01425	Rivet header, steel, for tubular rivets, up to D13mm

HAND TOOLS

Image for illustration only Original item may vary	Reference	Description
	74-01427	Cutter type Roller knife
	74-01429	Scissors, leather trimming shears, curved blade, 180mm
	74-01110	STAPLE GUN, for staples n.4 up to 14mm and nails
	74-01114	PIPE VICE, for pipes 10 to 60mm, 1/8 to 2", base 185 x 205mm
	74-01111	NUT DRIVER, n. 5.5, for hexagonal nut/bolt M3
	74-01112	NUT DRIVER, n.10 , for hexagonal nut/bolt M6
	On request	stainless steel, CENTERING PIN, for orthosis axis alignment

POWER TOOLS & EQUIPMENT

Image for illustration only Original item may vary	Reference	Description
	74-00340	Long arm plastic router MTF 100 F/FU/2,2/W, 450x900x1400mm
	74-00464	Long arm plastic router MTF 100 HV/FU, 600x1200x1300mm
	74-01020	(vacuum pump CR1000) TUBE, enveloping suction (1 small)
	74-01019	(vacuum pump CR1000) TUBE, enveloping suction (1 very-small)
	74-01025	SEWING MACHINE, ZIGZAG, electric operated, light/medium work
	74-01013	MANUAL SCREW PRESS arm 150mm, for inserting tubular rivets
	On request	VACUUM FORMING and LAMINATING STATION TZ-LR 2
	On request	VACUUM FORMING and LAMINATING STATION TZ-LR +
	74-01617	SAW, OCILLATING, pneumatic, for plaster/plastic, connect kit
	74-01171	(air compressor) SPIRALED HOSE, in.d. 6mmx5m

POWER TOOLS & EQUIPMENT

Image for illustration only Original item may vary	Reference	Description
	74-01182	BLOW GUN, aluminium, inner thread 1/4", for air compres.
	74-01189	DRILL BENCH TYPE, quick chuck 3 to 16mm, 230V, 550W
	74-01197	DRILL-FASTENER, cordless 14.4V, 2.4Ah bt, 13mm quick chuck
	74-01198	GRINDER BENCH, 2 wheels d.200mm, 230V, 450W
	74-01199	GRINDER mod 4300 with DISC, diam 350mm + LONG AXIS, on stand
	74-01205	LASER level and 2 D Quigo
	74-01642	HOSE, air pressure extension reel, 7m long, 1/4"
	74-01209	SAW, JIG, 230V, 720W.
	74-01210	SAW, OCILLATING, electric, for plaster & plastic 230V, 180W
	74-01222	WELDER "HOT AIR", LEISTER TRIAC "S", 230V



POWER TOOLS & EQUIPMENT

Image for illustration only Original item may vary	Reference	Description
	74-01223	WELDING "MIRROR" Diam 280 mm
	74-01224	WELDING MACHINE, electric, Primus 210 E
	74-01292	WELDER "HOT AIR", LEISTER Hot-Jet-S, small model, 230V
	74-01306	(workshop dust aspirator) FILTER BAG
	74-01307	DUST ASPIRATOR, for workshop without connecting kit
	74-01314	DRILL COLUMN TYPE quick chuck 3 to 16mm, 400V, 750W
	74-01351	WELDING IRON, electrical 150W, curved nozzle, for PP plastic
	74-01374	DUST CONTAINER, with blower duct system + hose, fixation kit
	74-01595	DOUBLE BELT GRINDER mod 553 on stand, belt 50 x 2500mm
	74-01598	LARGE BELT GRINDER mod 306, on stand, belt 250 x 2000mm

POWER TOOLS & EQUIPMENT

Image for illustration only Original item may vary	Reference	Description
	74-01621	ANGLE GRINDER 2400W, 230V , disc Diam 230mm
	74-01622	(angle grinder) BENCH STAND
	74-01630	DRILL, normal/impact, 230V, 660W, 13mm quick chuck, 2 speeds
	74-01641	HOSE, air pressure extension reel, 16m, 1/4"
	74-01808	EXTENSION CABLE, 15m reel, 3x1.5mm ² , wall fixation
	74-01809	CABLE, elect. flexible, 3 x1.5mm ² , outdoor use, black, meter
	74-01811	MULTIMETER digital TRMS Fluke 112
	74-01237	AIR COMPRESSOR UNIT, 380L/min, 10bar, tank 200L, 400V/3000W
	74-00716	SEWING MACHINE for leather, foot operated, lower arm 470mm
	74-01940	OVEN INFRARED 380V, with 2 teflon sheet

POWER TOOLS & EQUIPMENT

Image for illustration only Original item may vary	Reference	Description
	74-00346	VACCUM PUMP VA25 LITRE-Hr
	On request	SCALE, NUTRITION, 5 kg, grad. 2/5g, electro., platform s.s

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-00693	RIVET, iron, zinc plated, d. 3x20mm
	74-00696	RIVET, iron, zinc plated, d. 4x20mm
	74-00697	RIVET, copper, flat head, d. 3x20mm
	74-00698	RIVET, copper, flat head, d. 4x20mm
	74-01331	STRAP, velcro, PVC , with loop, brown, 300x25mm
	74-01332	STRAP, velcro, PVC , with loop, brown, 400x40mm
	74-01814	STUMP SOCK, trans-femoral, flat width 13cm, length 40cm - Pack of 10
	74-01815	STUMP SOCK, trans-femoral, flat width 13cm, length 50cm - Pack of 10
	74-01816	STUMP SOCK, trans-femoral, flat width 13cm, length 60cm - pack of 10
	74-01817	STUMP SOCK, trans-tibial, flat width 10cm, length 35cm - Pack of 10

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01818	STUMP SOCK, trans-tibial, flat width 10cm, length 45cm - Pack of 10
	74-01819	STUMP SOCK, trans-tibial, flat width 10cm, length 55cm - Pack of 10
	74-01172	(jig saw) SPARE BLADE, for plastics and aluminium
	74-01173	(oscillating saw) SPARE BLADE, for plaster & plastic
	74-01175	(grinder mod 4300) SELF-ADHESIVE ABRASIVE DISC 350mm Grit 24
	74-01180	BLADE BI-METAL, for hacksaw, 12 inches, 24 teeth
	74-01181	BLADE, for knife/cutter plastic 165mm, snap-off blade
	74-01190	DRILL "CONICAL" HSS, Diam 16 - 30 mm, L = 70mm.
	74-01191	DRILL "CONICAL" HSS, set of 3 Pces
	74-01192	DRILL "COUNTERSINK" HSS, 90deg, Diam 2 up to 8mm

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01193	DRILL "COUNTERSINK" HSS, 90deg, Diam 3.2 up to 16.5mm
	74-01194	DRILL BIT SET, HSS Polished flutes, 25pcs 1-13mm, 0.5mm incr.
	74-01195	DRILL BIT SET, masonry, SDS diam 4/5/6/8/10/12 mm
	74-01196	DRILL BIT SET, stainless steel, black oxide 24pcs, 1-10.5mm
	74-01207	REINFORCED PVC HOSE, in.d. 8mm, for compres. air
	74-01298	SREW, Countersunk head bolt, thread, HSS, DIN 7991, M10X60
	74-01301	NUT, LOCK, zing plated, M4, Serpress
	74-01308	(dust aspirator) HOSE CLAMP from 87 to 112mm diam
	74-01309	(dust aspirator) FLEXIBLE HOSE, diam 102mm x 2,5m
	74-01310	(dust aspirator) ANTI-VIBRATING hoseclamp wall fix. d.100mm

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01316	STRAP, velcro, 25mmX46M, Hook&Loop
	74-01543	ELECTRODE, welding, coated, rutile, 2.5 mm
	74-01544	ELECTRODE, welding, coated, rutile, 3.2mm
	74-01545	ELECTRODE, welding, for S.STEEL,coated, V2A, 2.5mm
	74-01546	ELECTRODE, welding, for S.STEEL,coated, V2A, 3.2mm
	74-01556	MASK, dust protecting, coarse dust, disposable
	74-01580	VALVE, BALL, 1/2"
	74-01581	HOSE BEARING, 1/2" x 16mm
	74-01582	BRONZE CROSS COUPLING, 1/2"
	74-01583	PLUG, male, 1/2"

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01584	DOUBLE CONNECTION, 1/2"
	74-01585	HOSE, reenforced PVC, Int Diam 16mm
	74-01586	TEFLON, insulating tape, 12mmx10m
	74-01587	HOSE CLAMP, Diam 15-24mm
	74-01588	SANDING BELT, 1750x150mm, Grit 24 for machine OOMA-MARP3000
	74-01589	SANDING BELT, 1750x150mm, Grit 60 for machine OOMA-MARP3000
	74-01593	(grinder mod 4300) SELF-ADHESIVE ABRASIVE DISC 350mm Grit 60
	74-01594	(grinder mod 4300) DISC ALUMINIUM diam 350mm
	74-01596	(double belt grinder mod 553) BELT, 50 x 2500mm, grit 24
	74-01597	(double belt grinder mod 553) BELT, 50 x 2500mm, grit 60

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01599	(large belt grinder mod 306) BELT, 250 x 2000, grit 24
	74-01600	(large belt grinder mod 306) BELT, 250 x 2000, grit 60
	74-01602	CYLINDRICAL HELICAL CUTTER, HSS, Diam 50mm
	74-01603	SPIRAL HELICAL CUTTER, HSS, Diam 27mm
	74-01605	WIRE WHEEL, Diam 150mm, Width 30mm
	74-01606	BUFFING COMPOUND, for plastic, steel and stainless steel
	74-01607	LAPPING WHEEL, diam. 200mm, diam. bore 16mm
	74-01609	SANDING STRIP ROLL, grit 24, 100mm wide
	74-01610	SANDING STRIP ROLL, grit 60, 100mm wide
	74-01614	(dust aspirator) ADHESIVE BAND with rubber mastic 5cmx2.5m

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01619	(jig saw) SPARE BLADE, for wood, till 60 mm cut
	74-01620	(jig saw) SPARE BLADE, for metal, till 6 mm cut
	74-01623	(angle grinder) CUTTING DISC, 230mm, for metal
	74-01624	(angle grinder) GRINDING DISC, 230mm, for stone
	74-01625	(angle grinder) GRINDING DISC, 230mm, for metal
	74-01626	(bench grinder) SPARE WHEEL, diam 150mm, grit 60
	74-01627	(bench grinder) SPARE WHEEL, diam 150mm, grit 36
	74-01640	AIR FILTER, for air compressor 380L/min, 10bar
	74-01643	QUICK COUPLING, female part, 1/4" inner thread, compr. air
	74-01644	QUICK COUPLING, male part, 1/4" outside thread, compr. air







HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01645	QUICK COUPLING, male part, 1/4" inside thread, compr. air
	74-01646	QUICK COUPLING, male part, tail for hose in d.8mm, comp. air
	74-01647	HOSE CONNECTOR, 1/4" outer thread to tail 8mm, comp. air
	74-01648	HOSE CONNECTOR, 1/4" inner thread to tail 8mm, comp. air
	74-01649	HOSE JUNCTION, for compressed air hose in.d. 8mm
	74-01650	T SHAPE HOSE JUNCTION for hose in. d.8mm, comp. air
	74-01651	Y SHAPE HOSE JUNCTION for hose in. d.8mm, comp. air
	74-01652	REDUCTOR, outer thread 1/2", inner thread 1/4", compres. air
	74-01653	DOUBLE CONNECTION, outer threads 1/4" - 1/4", compres. air
	74-01654	DOUBLE CONNECTION, outer threads 1/2" - 1/4", compres. air




HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01655	DOUBLE CONNECTION, inner thread 1/4" - 1/4", compres. air
	74-01656	Y SHAPE CONNECTION, outer thread 1/4", for compres. air
	74-01657	CONNECTOR, bend 90deg, outer thread 1/4", for compres. air
	74-01658	CONNECTOR, bend 90deg, inner thread 1/4", for compres. air
	74-01659	T SHAPE CONNECTOR, outer thread 1/4", for compres. air
	74-01660	T SHAPE CONNECTOR, inner thread 1/4", for compres. air
	74-01661	CROSS CONNECTOR, 3 inner + 1 outer threads 1/4*, comp. air
	74-01662	BALL VALVE, 1/4 turn, inner threads 1/4", for compres. air
	74-01664	BAND CLIP, for hose pipe, high torque, 11-17mm
	74-01665	NUT, hexagonal M10, HSS A2, DIN 980

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01666	WASHER, FLAT, HSSl, dia. 6.5X25mmX1.5mm
	74-01667	SCREW, head bolt, HSS, DIN 931, M6 x 80
	74-01668	SCREW, head bolt, HSS, DIN 931, M6 x 70
	74-01669	SCREW, Countersunk, HSSM6X5
	74-01670	NUT torque , hexagonal M10, HSS A2, DIN 1587
	74-01671	BOLT, stainless steel, hexagonal head, M10 x 80mm
	74-01672	BOLT, stainless steel, hexagonal head, M10 x 100mm
	74-01673	SREW,Countersunk head bolt, thread, HSS,DIN 7991, M8X60
	74-01674	WASHER, Flat, HSS, M10 M15 DIN 9021 diam 44
	74-01675	WASHER, Flat, HSS, M10.5 DIN9021 diam 40











HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01676	WASHER, Flat, HSS, M10.55 dia25
	74-01677	SCREW, Countersunk head bolt, thread, HSS, DIN 7991 M10x50
	74-01678	WASHER, Flat, HSS, M15 dia54
	74-01679	NUT, hexagonal M10, HSS A2, DIN 934
	74-01680	SCREW, Countersunk head bolt, HSS A2, DIN 912, M10 x 90
	74-01681	BOLT, stainless steel, socket head, M10 x 80mm
	74-01682	SCREW, Countersunk head bolt, HSS A2, DIN 912, M8 x 80
	74-01683	WASHER, Spring, HSS, M10 DIN 127B
	74-01684	WASHER, FLAT, HSS, DIN125A, dia 20X10.5X2mm DIN125A
	74-01685	WASHER, Spring, HSS, M8 DIN 127B











HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01686	WASHER, FLAT, HSS, DIN125A,dia 8.4X16X1.6mm
	74-01687	NUT, torque hexagonal M10, HSS A2, DIN 985
	74-01688	WASHER, FLAT IRON, zing plated, diam. 10x25mm, thickness 2mm
	74-01689	WASHER, EXTERNAL TOOTH LOCK, zing plated, diam. 10mm
	74-01690	NUT, HEXAGONAL, zing plated, M10
	74-01691	WASHER, FLAT IRON, diam. 6.4x16mm, thickness 1.6mm
	74-01692	WASHER, EXTERNAL TOOTH LOCK, zing plated, diam. 6mm
	74-01694	SCREW, MACHINE, flat head slotted, zing plated, M4x12mm
	74-01695	BOLT, stainless steel, socket head, M10 x 90mm
	74-01696	Rib spring washer, Hss, Diam . 105mm

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01697	Rib spring washer, Hss, Diam. 84mm
	74-01725	HAND REAMER SET, HSS, 10 pieces d. 5 to 16mm
	74-01726	MACHINING TAP, hand use, HSS for Sainless Steel, M3
	74-01727	MACHINING TAP, hand use, HSS for Sainless Steel, M4
	74-01728	MACHINING TAP, hand use, HSS for Sainless Steel, M5
	74-01729	MACHINING TAP, hand use, HSS for Sainless Steel, M6
	74-01730	MACHINING TAP, hand use, HSS for Sainless Steel, M8
	74-01731	TAP SET, 3 pces, by hand, HSS, M8
	74-01732	TAP SET, 3 pces, by hand, HSS, M10
	74-01733	TAP SET, 3 pces, by hand, HSS, M12

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01740	DIE, by hand, HSS, M3
	74-01741	DIE, by hand, HSS, M4
	74-01742	DIE, by hand, HSS, M5
	74-01743	DIE, by hand, HSS, M6
	74-01744	DIE, by hand, HSS, M8
	74-01745	DIE, by hand, HSS, M10
	74-01746	DIE, by hand, HSS, M12
	74-01747	UNIVERSAL OIL, for threading and drilling, flask
	74-01751	ROPE, for plumb bob, diam 1.5mm, yellow colour
	74-01754	PLASTER OF PARIS, POWDER

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01759	DRILL BIT, High Speed Steel, Polished flutes, diam.1mm
	74-01760	DRILL BIT, High Speed Steel, Polished flutes, diam.1.5mm
	74-01761	DRILL BIT, High Speed Steel, Polished flutes, diam.2mm
	74-01762	DRILL BIT, High Speed Steel, Polished flutes, diam.2.5mm
	74-01763	DRILL BIT, High Speed Steel, Polished flutes, diam.4.5mm
	74-01764	DRILL BIT, High Speed Steel, Polished flutes, diam.4.8mm
	74-01765	DRILL BIT, High Speed Steel, Polished flutes, diam.5.5mm
	74-01766	DRILL BIT, High Speed Steel, Polished flutes, diam.6mm
	74-01767	DRILL BIT, High Speed Steel, Polished flutes, diam.6.5mm
	74-01768	DRILL BIT, High Speed Steel, Polished flutes, diam.7mm











HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01769	DRILL BIT, High Speed Steel, Polished flutes, diam.7.5mm
	74-01770	DRILL BIT, High Speed Steel, Polished flutes, diam.8mm
	74-01771	DRILL BIT, High Speed Steel, Polished flutes, diam.8.5mm
	74-01772	DRILL BIT, High Speed Steel, Polished flutes, diam.9mm
	74-01773	DRILL BIT, High Speed Steel, Polished flutes, diam.9.5mm
	74-01774	DRILL BIT, High Speed Steel, Polished flutes, diam.10mm
	74-01775	DRILL BIT, High Speed Steel, Polished flutes, diam.10.5mm
	74-01776	DRILL BIT, High Speed Steel, Polished flutes, diam.11mm
	74-01777	DRILL BIT, High Speed Steel, Polished flutes, diam.11.5mm
	74-01778	DRILL BIT, High Speed Steel, Polished flutes, diam.12mm

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01779	DRILL BIT, High Speed Steel, Polished flutes, diam.12.5mm
	74-01780	DRILL BIT, High Speed Steel, Polished flutes, diam.13mm
	74-01781	DRILL BIT, High Speed Steel, Polished flutes, diam.13.5mm
	74-01782	DRILL BIT, High Speed Steel, Polished flutes, diam.14mm
	74-01783	DRILL BIT, High Speed Steel, Polished flutes, diam.14.5mm
	74-01784	DRILL BIT, High Speed Steel, Polished flutes, diam.15mm
	74-01785	DRILL BIT, High Speed Steel, Polished flutes, diam.15.5mm
	74-01786	DRILL BIT, High Speed Steel, Polished flutes, diam.16mm
	74-01787	DRILL BIT SET, HSS Polished flutes, 19pcs 1-10mm, 0.5mm incr.
	74-01788	DRILL BIT, stainless steel, 3.2mm, black oxide

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01789	DRILL BIT, stainless steel, 4.2mm, black oxide
	74-01790	DRILL BIT, stainless steel, 5mm, black oxide
	74-01791	DRILL BIT masonry SDS diam 6 x 150 mm
	74-01792	DRILL BIT masonry SDS diam 10 x 200 mm
	74-01793	DRILL BIT masonry SDS diam 12 x 200 mm
	74-01794	DRILL BIT masonry SDS diam 14 x 250 mm
	74-01795	DRILL BIT masonry SDS diam 16 x 250 mm
	74-01796	DRILL BIT masonry SDS diam 18 x 250 mm
	74-01797	DRILL BIT masonry SDS diam 20 x 400 mm
	74-01798	DRILL BIT SET, for wood, 3/4/5/6/7/8/9/10 mm











HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01799	(cutting device) SPARE KNIFES, length sup. 149 x inf. 120mm
	74-01800	(cutting device) SPARE KNIFES, length sup. 173 x inf. 150mm
	74-01801	(cutting device) SPARE KNIFES, length sup. 211 x inf. 180mm
	74-01802	(cutting device) SPARE KNIFES, length sup. 186 x inf. 175mm
	74-01810	TAPE, insulating, plastic, 19mm x 30m, black
	74-01813	GREASE, Motorex 190 EP, for coppet knee joint bushing
	74-01863	(bench grinder) SPARE WHEEL, diam 200mm, grit 36
	74-01864	(bench grinder) SPARE WHEEL, diam 200mm, grit 60
	74-01693	NUT, HEXAGONAL, zing plated, M6
	On request	BANDAGE ADHESIVE, textile, 5cmx10m, non elastic

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	On request	BANDAGE, ELASTIC, 08 cm x 5 m, constraining
	On request	BANDAGE, ELASTIC, 10 cm x 5 m, constraining
	On request	BANDAGE, ELASTIC, ADHESIVE , textile 6cmx2.5m
	On request	BANDAGE, ELASTIC, ADHESIVE , textile 10cmx2.5m
	On request	BANDAGE PADDING, for POP, 10cmx2.7m, viscose 100%
	On request	BANDAGE PADDING, for POP, 15cmx2.7m, viscose 100%
	On request	BANDAGE, TRIANGULAR, 136 x 96 x 96 cm
	On request	COTTON, UNBLEACHED, 500g, carded (to be used with POP)
	On request	GAUZE, 100x0.9m, 20 threads, for compresses, roll
	On request	TAPE, ADHESIVE DRESSING, 06 cm x 5 m, roll

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	On request	TAPE, ADHESIVE DRESSING, 08 cm x 5 m, roll
	On request	TAPE, ADHESIVE PAPER, 2.5 cm x 10 m, roll
	On request	TAPE, ADHESIVE PAPER, 5 cm x 10 m, roll
	74-00699	TUBULAR STOCKINETTE, cotton, 100mm, roll of 25m
	74-00700	TUBULAR STOCKINETTE, cotton, 120mm, roll of 25m
	74-00701	TUBULAR STOCKINETTE, cotton, 150mm, roll of 25m
	74-00702	TUBULAR STOCKINETTE, cotton, 200mm, roll of 25m
	74-00703	TUBULAR STOCKINETTE, cotton, 250mm, roll of 25m
	74-00704	TUBULAR STOCKINETTE, cotton, 300mm, roll of 25m
	74-00705	TUBULAR STOCKINETTE, cotton, 350mm, roll of 25m

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-00706	TUBULAR STOCKINETTE, cotton, 400mm, roll of 25m
	74-00707	TUBULAR STOCKINETTE, cotton, 60mm, roll of 25m
	74-00708	TUBULAR STOCKINETTE, cotton, 80mm, roll of 25m
	74-00714	BANDAGE, ELASTIC, 12 cm x 5 m, constraining/compressive
	74-00749	DRILL BIT, High Speed Steel, Polished flutes, diam.3mm
	74-00750	DRILL BIT, High Speed Steel, Polished flutes, diam.3.2mm
	74-00751	DRILL BIT, High Speed Steel, Polished flutes, diam.3.5mm
	74-00752	DRILL BIT, High Speed Steel, Polished flutes, diam.4mm
	74-00753	DRILL BIT, High Speed Steel, Polished flutes, diam.5mm
	74-00782	LOOP, 25mm










HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-00783	LOOP, 35mm
	74-00788	LADY STOCKING, nylon, for cast taking and thermoforming, pair
	74-00790	SPARE BLADE, for "Stanley Surform" half round, blade 250mm
	74-00792	BANDAGE, PLASTER OF PARIS, 10 cm x 3 m
	74-00793	BANDAGE, PLASTER OF PARIS, 15 cm x 3 m
	74-00794	BANDAGE, PLASTER OF PARIS, 20 cm x 3 m
	74-00804	POLISHING WHEEL in cotton, diam 250mm
	74-00846	PENCIL, blue, indelible, for marking on moist surface
	74-00919	PENCIL, red, soft lead pencil, for marking on GLASS
	74-00849	ROTARY RASP HEAD, d.30mm, length 105mm

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-00850	ROTARY RASP HEAD, d.35mm, length 55mm
	74-00851	ROTARY RASP HEAD, d.45mm, length 110mm
	74-00855	SANDING CONE GRIT 150, for arbor 064020-016
	74-00856	SANDING CONE GRIT 50, for arbor 064020-016
	74-00858	SANDING DRUM WITH CONICAL HOLE Diam 72,5 mm x 200 mm
	74-00860	SANDING SLEEVE, grit 50 for 064075 - 200
	74-00861	SANDING SLEEVE, grit 80, for 064075 - 200
	74-00862	SANDING STRIP ROLL, grit 40, 100mm wide
	74-00863	SANDING STRIP ROLL, grit 80, 100mm wide
	74-00864	SANDING STRIP ROLL, grit 120, 100mm wide











HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-00867	(zigzag sewing machine) NEEDLES, system 438KK, size 100
	74-00868	(zigzag sewing machine) NEEDLES, system 438KK, size 110
	74-00906	(zigzag sewing machine) THREAD, black, serafil, strength 40
	74-00907	(zigzag sewing machine) THREAD, white, serafil, strength 40
	74-00913	RIVET, tubular, brass nickel plated, 13x12mm
	74-00914	RIVET, tubular, iron nickel plated, 8x9mm
	74-00916	FLAT WASHER BRASS, Ø 13 x 3.1 thickness 0.5 mm
	74-00917	FLAT WASHER BRASS, Ø 16 x 4.1 thickness 0.5 mm
	74-01322	BUCKLE, plastic & insertable, 25mm
	74-01366	STRAP, velcro, 30mm, Hook.

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01367	STRAP, velcro, 30mm, Loop/Velvet.
	74-01414	Strap, perlon webbing, 25mm - Roll of 50m
	74-01415	Strap, elastic, 25mm - Roll of 25m
	74-01416	Strap, elastic, 35mm - Roll of 25m
	74-01417	Strap, prostesis, 25mm - Roll of 50m
	74-01418	Strap, prostesis, 35mm - Roll of 50m
	74-01419	Strap, polyester, black, 40mm
	74-01873	BUCKLE, plastic, 30mm
	74-01420	Buckle, plastic & insertable, 40mm
	74-01421	Ring, round, 25mm - Pac of 100pcs





HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01422	Ring, halfround, 25mm - Pack of 100pcs
	74-01426	Rivet, aluminium, flat head, D3x20mm - Pack of 1'000pcs
	74-01428	Spare blade for cutter type roller knife
	74-01430	Talc powder, conform to pharmacopoeia (asbestos fibres free)
	74-01431	Glue, neoprene contact, 4kg drum of 4kg
	74-01432	Solvent thinner, drum of 5L for glue neoprene contact
	74-01435	Needles, size 120 for leather sewing machine (74-00716) - Pack of 10pcs
	74-01436	Needles, size 140 for leather sewing machine (74-00716) - Pack of 10pcs
	74-01437	Thread, black, rasant, strength 50 for leather sewing machine (74-00716)
	74-01438	Thread, white, rasant, strength 50 for leather sewing machine (74-00716)

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	74-01439	Lubricating oil, tin of 1dl for leather sewing machine (74-00716)
	On request	BUCKLE, plastic, 30mm
	74-01109	GLUE, Loctite n.243, blue thread lock, btl. 50ml
	74-01108	GLUE, Loctite 290, thread lock, btl. 50ml
	74-01113	STAPLES, for staple gun, n.4, 12mm
	74-00713	BANDAGE, ELASTIC, 10 cm x 5 m, constraining/compressive
	74-00715	BANDAGE, ELASTIC, 15 cm x 5 m, constraining/compressive
	74-01950	Examination glove, latex, non-steril, M (7-8) - Pack of 100pcs
	74-01949	Examination glove, latex, non-steril, XL (10-11) - Pack of 90pcs
	On request	Examination glove, latex, non-steril, L - Pack of 100pcs

HARDWARE & CONSUMABLES

Image for illustration only Original item may vary	Reference	Description
	On request	SET, PLASTER OF PARIS, 10-20 plaster casts
	74-00739	DRILL "FORSTNER BIT" Diam 22 mm
	74-00740	DRILL "FORSTNER BIT" Diam 25 mm
	74-00741	DRILL "FORSTNER BIT" Diam 30 mm











SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-00805	SPARE BLADE, for "Stanley Surform" round file 250mm
	74-00807	SPARE BLADE, type C, brass + plastics, for deburring tool
	74-00883	Quickchange deburring tool, spare blades for holes in steel
	74-00885	SPARE BLADE, type A, aluminium + steel, for deburring tool
	74-00901	SPARE BLADE, for "Stanley Surform" std flat file 250mm
	74-00903	ROLL OF TEFLON, 1m wide, 10 meter long
	74-01618	(oscillating saw) SPARE BLADE, for plaster & plastic large
	On request	(Four CR 5000G) Quit joint/ porte =10 mètres joint & 1 cart
	74-01014	(zigzag sewing machine) SHUTTLE, without spool
	74-00936	(oven CR5000) TIMER'S KNOB

SPARE PARTS & ACCESSORIES

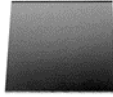Image for illustration only Original item may vary	Reference	Description
	74-01176	(welder triac S) HEATING ELEMENT, type 33, 1550W
	74-01177	(welder triac S) HOLDING NOZZLE, diam. 32mm, mod. 100.303
	74-01178	(welding "hot air") SPARE CARBON BRUSH
	74-01179	(welding "hot air") WELDING NOZZLE, Diam 4mm
	74-01293	(welder Hot-Jet-S) HOLDING NOZZLE, diam. 22mm, mod. 107.144
	74-01294	GROUND CONNECTOR, for electric arc welding machine, 4m cable
	74-01313	CHUCK, quick acting, for bench drilling machine, 0.5 to 13mm
	74-01538	(welder Hot-Jet-S) V SHAPED WELDING NOZZLE, triangle 3.7x5.7mm
	74-01539	(welder Hot-Jet-S) V SHAPED WELDING NOZZLE, triangle, 5x7mm
	74-01540	(welder Hot-Jet-S) HEATING ELEMENT, type 33H, 435W

SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01541	(welder leister Hot-Jet-S) SPARE CARBON BRUSH
	74-01542	(welding iron) VOLTAGE STABILIZER, 500W, 230V
	74-01547	(welding machine) RADIANCE PROTECTIVE SHIELD
	74-01548	(welding machine) PROTECTIVE GLASSES "ATHERMAL"
	74-01549	(welding machine) TRANSPARENT PROTECTIVE GLASS
	74-01550	APRON, for welder personal protection
	74-01551	GLOVES, for welder personal protection, one pair
	74-01553	ELECTRODE HOLDER, for electric arc welding machine, 5m cable
	74-01555	GLASSES, clear rigid plastic, eyes front + sides protection
	74-01557	GLOVE, protection, heavy duty leather, size 8











SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01558	GLOVE, protection, heavy duty leather, size 11
	74-01559	(oven CR5000) CONNECTION PLATE, NC5933 - 31
	74-01560	(oven CR5000) THERMICAL FUSE, ENP 220
	74-01561	(oven CR5000) TIMER, 120 Min.
	74-01562	(oven CR5000) SWITCH, SP 400 E 1 16 (4) A /250V
	74-01563	(oven CR5000) CONTACTOR, KM DILEM-10 (ex: K9-10 and ex: relais TRP614)
	74-01564	(oven CR5000) LAMP, 40W, 240V, 550C
	74-01565	(oven CR5000) BULB, 40W, 240V, 300C, E14
	74-01566	(oven CR5000C) BULB, SMALL, 15W/300C
	74-01567	(oven CR5000) BUZZER, BE 220 / 240 / 3001

SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01568	(oven CR5000) BLADE FAN + ENGINE
	74-01569	(oven CR5000C) ROUND / CIRCULAR RESISTANCE
	74-01570	(oven CR5000G) TUBE HEATING ELEMENT, 750W/230V, L= 840mm
	74-01571	(oven CR5000) FEELER for thermoregulator TLK48
	74-01572	(oven CR5000) FUSE, DO 1 GL 2 Amp.
	74-01573	(oven CR5000) FUSE, DO 1 GL 10 Amp.
	74-01574	(oven CR5000) CLAMP, 06 8 -18 12 857
	74-01575	(oven CR5000G) TUBE HEATING ELEMENT, 750W/230V, L= 980mm
	74-01576	(oven CR5000G) THERMOREGULATOR, TLK 48 (ex:E5CX, ex:THP48)
	74-01577	(oven CR5500) LUMINOUS SWITCH

SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01579	(oven CR) THERMOMETER
	74-01590	(for all grinder models) SAFETY FOOT SWITCH
	74-01591	(grinder mod 3300,4300,553) ELECTRICAL PANEL
	74-01592	(large belt grinder mod 306) ELECTRICAL PANEL
	74-01601	ADAPTOR, M16 male-male
	74-01604	THREADED HOLDER, for edge cutter
	74-01608	CONICAL SANDING ARBOR
	74-01611	(dust aspirator) GALVA REDUCTOR 150 to 100mm diam
	74-01612	(dust aspirator) GALVA STRAIGHT CONNECTOR diam 100mm
	74-01613	(dust aspirator) T-SHAPED OPEN-CLOSE VALVE, diam 100mm

SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01615	(dust aspirator) FILTER DUST CRATRIDGES
	74-01616	(dust aspirator) T-SHAPED CONNECTOR, diam 100mm
	74-01628	(bench grinder) STAND
	74-01629	(bench grinder) DRESSING APPARATUS, with 8 wheels diam. 50mm
	74-01631	(drill impact) DRILL CHUCK KEY TYPE, 1.5-13mm + SDS adapter
	74-01632	(drill bench type) V-BELT, reference number B31 and B 34.5
	74-01633	(drill bench type) DRILL DRIFT PLAIN, din317, m.c 2
	74-01634	(drill column type) V-BELT, reference number A22 and A26
	74-01635	CHUCK, quick acting, for column drilling machine, 3 to 16mm
	74-01636	(drill column type) DRILL DRIFT PLAIN, din317, m.c 3

SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01637	(drilling machine) VICE for column drilling machine
	74-01638	CONE MANDREL, type CM2/16, for drilling milling machine
	74-01639	CONE MANDREL, type CM2/18, for drilling milling machine
	74-01804	SWIVEL BASE, for bench vice 140mm
	74-01861	BATTERY, 14.4V, 2.4Ah, for cordless drill-fastener
	74-01862	BATTERY CHARGER, fast, for cordless driller batt.14.4V,2.4Ah
	74-01865	(workshop dust aspirator) DUST BAG, roll of 10 bags
	74-00461	ABRASIVE FLAP WHEEL
	74-00712	EARS PROTECTORS, snr 23dB (protect. 23dB inf. ambient noise)
	74-00744	GLOVE, heat protection, isothermal, pair, size 10





SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-00784	(cutting device) STEEL STAND with two wheels
	74-00838	SPARE RETAINER SPRING, for pneumatic chipping hammer
	74-00852	SET OF 5 SPARE RUBBER STRIPS for abrasive flap wheel
	74-00854	GLASSES INTEGRAL PROTECTION, clear rigid plastic, elas. strap
	74-00857	SANDING DRUM, Diam 90mm, Length 100mm
	74-00859	SANDING DRUM, Diam 60mm, Length 100mm
	74-00875	SPARE WHEELS, for heavy duty pipe cutter 1/8" to 2"
	74-00904	ADAPTOR, from 5/8 female to M16 male
	74-00905	ADAPTOR, from M16 female to 5/8 male
	74-01433	Spool for leather sewing machine (74-00716)

SPARE PARTS & ACCESSORIES

Image for illustration only Original item may vary	Reference	Description
	74-01434	Shuttle without spool for leather sewing machine (74-00716)
	On request	GLOVE, nylon full palm PU coated, black, size 8 or M, pair
	On request	GLOVE, nylon full palm PU coated, black, size 11 or XXL, pair
	On request	OVEN INFRARED Teflon Coating Sheet
	On request	VACUUM FORMING STATION Distributor Terminal











MOBILITY AIDS

Image for illustration only Original item may vary	Reference	Description
	On request	CRUTCH, elbow, high 74/97 cm
	On request	CRUTCH, child, height 47/ 74cm
	On request	CRUTCH, AXILLARY, adult
	On request	RUBBER TIP, d.19mm for elbow & axillary crutches, adult&child











PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01012	MIRROR, square, mobile on wheels
	74-01022	PARALLEL BARS, 4m, adjustable height and width
	74-00002	TABLE, treatment basic, not foldable
	On request	BASKET BALL
	On request	VOLLEY BALL
	74-01474	Rope for pullytherapy, nylon polyamide, diam 6mm, braided, M
	On request	INSTRUCTIONAL CHART, upper limb FR
	On request	INSTRUCTIONAL CHART, upper limb B
	On request	INSTRUCTIONAL CHART, lower limb FR
	On request	INSTRUCTIONAL CHART, lower limb B





PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	On request	INSTRUCTIONAL CHART, muscular system B
	On request	INSTRUCTIONAL CHART, muscular system FR
	On request	INSTRUCTIONAL CHART, nervous system FR
	On request	INSTRUCTIONAL CHART, nervous system B
	On request	INSTRUCTIONAL CHART, "human skeleton" B
	On request	INSTRUCTIONAL CHART, "human skeleton" FR
	On request	INSTRUCTIONAL CHART, "spine" B
	On request	INSTRUCTIONAL CHART, "spine" FR
	On request	INSTRUCTIONAL CHART, vascular system B
	On request	INSTRUCTIONAL CHART, vascular system FR











PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	On request	SKELETON, Muscle Arm
	74-01455	Bench quadriceps and accessories
	74-01457	Mini pedalling trainer PRO
	74-01456	Bicycle, fixed, mechanical
	On request	Standing Frame, Ferrox Raffaello - Kenia
	On request	Standing Frame Rupiani LV XP, adjustable height, with table
	74-01463	Wall Bars, wooden
	On request	GYMNASTIC BALL, 120 cm diameter
	74-01483	Gymnastic ball, 45 cm diameter
	74-01484	Gymnastic ball, 65 cm diameter

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01485	Gymnastic ball, 75 cm diameter
	74-01486	Gymnastic ball, 85 cm diameter
	74-01487	Gymnastic ball, 95 cm diameter
	74-01488	Pump with support for all balls+needle+end connector
	74-01489	Gymnastic mat, 180cm x 60cm x 1-1,5cm
	74-01490	Gymnastic mat, 200cm x 100cm x 2,5cm
	74-01491	Gymnastic mat, Thickness more than 3cm
	74-01464	Ankle boot for pulleytherapy, vinyl
	74-01465	Sandbag for pulleytherapy, 0.5kg + fixation ring
	74-01466	Sandbag for pulleytherapy, 1kg + fixation ring

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01467	Sandbag for pulleytherapy, 2kg + fixation ring
	74-01468	Sandbag for pulleytherapy, 3kg + fixation ring
	74-01469	Sandbag for pulleytherapy, 4kg + fixation ring
	74-01470	Sandbag for pulleytherapy, 5kg + fixation ring
	74-01471	Sandbag for pulleytherapy, set of 6 (0.5kg to 5kg) + fixation ring
	74-01472	Grid cage for pullytherapy, pullytherapy
	74-01473	Grid cage for pulleytherapy, Ferrox
	74-01475	handle for pullytherapy whith one ring
	74-01476	Hook "S" for pullytherapy, size 5
	74-01477	Hook "S" for pullytherapy, size 7

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01478	Hook "S" for pulletherapy, snap
	74-01479	SLING, fixation lower limb (vinyl)
	74-01480	Sling, fixation trunk (vinyl)
	74-01481	Sling, suspension, lower limb (vinyl)
	74-01482	Sling, suspension, trunk-pelvis (vinyl)
	74-01492	Balance board, diam 40cm
	On request	BALANCE EXERCISE TOP, rectangular, 40x50cm
	74-01493	Cushion, cylindric, diameter 15cm, L 40-50cm
	74-01494	Cushion, half-roll; 50x20x10cm
	74-01495	Cushion, massage; 45x20x7cm

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01496	Cushion, neck, hourglass type
	74-01497	Cushion, wedge shape 50x30x15cm
	74-01498	Cushion, wedge shape 50x40x30cm
	74-01499	Digiflex with 5 springs - set of 5 strengths
	74-01500	Dumb bell, 1kg - pair
	74-01501	Dumb bell, 2kg - pair
	74-01502	Dumb bell, 3kg - pair
	74-01503	Dumb bell, 4kg - pair
	74-01504	Dumb bell, 5kg - pair
	74-01506	Eggerciser - set of 5 strengths

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01507	Eggerciser, soft
	74-01505	Eggerciser, medium
	74-01508	Eggerciser, strong
	74-01509	Eggerciser, ultra strong
	74-01510	Medecine ball, 1-1.5kg
	74-01511	Medecine ball, 2kg
	74-01512	Medecine ball, 3kg
	74-01513	Medecine ball, 4kg
	74-01514	Medecine ball, 5kg
	74-01515	Power web junior, rubber - set of 3

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01516	Putty, extra strong 450g
	74-01517	Putty, medium 450g
	74-01518	Putty, soft 450g
	74-01519	Putty, strong 450g
	On request	ROLL, for exercise diam.15cm L50cm
	On request	ROLL, for exercise diam.30cm L100cm
	74-01520	Shoulder wheel, steel, diam 100cm
	74-01521	Theraband, extra strong, 5.5m
	74-01522	Theraband, medium, 5.5m
	74-01523	Theraband, strong, 5.5m











PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01524	Weight strap, 0.5kg
	74-01525	Weight strap, 1kg
	74-01526	Weight strap, 2kg
	74-01527	Weight strap, 3kg
	74-01528	Weight strap, 4kg
	74-01529	Weight strap, 5kg
	74-01531	Cool pack, 25x15cm
	74-01532	Cool pack, 19x28cm
	74-01533	Cool pack, 28x36cm
	74-01534	Cool pack, 28x53cm

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01535	Cool pack, long cervical, 58cm, vinyl cover
	On request	HOT PACK, 45x25cm
	On request	HOT PACK, cervical;60cm
	74-01530	Cool pack, Ice bag PVC diam 28
	74-01451	Goniometre, for finger articulations
	74-01454	Scale for adults (bathroom type), mecan,0-150kg/0.1kg
	74-01453	Hammer percussion, 20cm
	74-01452	Wall viewer X-ray, 50 x 78, complete
	74-01458	Ankle support for bench quadriceps 74-01455
	74-01440	Skeleton, human with stand

PHYSICAL REHABILITATION AIDS

Image for illustration only Original item may vary	Reference	Description
	74-01441	Skeleton, human, mini, with stand
	74-01442	Skeleton, spine, with pelvis, on stand
	74-01443	Skeleton, foot
	74-01444	Skeleton, hip joint
	74-01445	Skeleton, knee joint
	74-01446	Skeleton, Muscle Leg
	74-01447	Skeleton, Functional Shoulder
	74-01448	Skeleton, functional elbow
	On request	Standing Frame Ferrox Canaletto Pro, hydraulic, manual
	74-01876	(Pulley) PULLEY, with hook snap

GLOSSARY


ICRC:	International Committee of the Red Cross
PRP:	Physical Rehabilitation Programme
LMIC:	Low-and middle-income countries
R&D:	Research and Development
EPFL:	Ecole Polytechnique Fédérale de Lausanne
ISO:	International Organization for Standardization
TTA:	Transtibial Adult
TTC:	Transtibial Child
TFA:	Transfemoral Adult
TFC:	Transfemoral Child
LS:	Long Stump
SACH:	Solid ankle, cushion heel
P&O:	Prosthetist-Orthotist
PP:	Polypropylene
POM:	Polyoxymethylene
TPE:	Thermoplastic Elastomere
UV:	Ultraviolet
°C:	Degrees Celsius
°F:	Degrees Fahrenheit
kg:	Kilogramm

CONTACT


REHAB'IMPULSE

FONDATION ALFASET

Rue des Terreaux 48
CH-2300 La Chaux-de-Fonds
Switzerland

 +41 32 967 96 50

 contact@rehabimpulse.org

 www.rehabimpulse.org





Rehab
Impulse[™] Promoting
Independent
Lifestyle